# Espelho Clínico Metálico

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ECM 5 - Ø 24 mm  
ECM 9 - Ø 40 mm  
ECM 10 - Ø 55 mm

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## USE INSTRUCTIONS

ECM 5 - Ø 24 mm  
ECM 9 - Ø 40 mm  
ECM 10 - Ø 55 mm

**INDICATION:** Manual instrument indicated to assist the professional in visualization the oral cavity for the diagnosis.

**INSTRUCTIONS:** Take the mirror to the oral cavity and perform the procedure.

**CLEANING:** The organic matter removal must be performed in ultrasonic cleaner, using enzymatic soap (minimum three enzymes) in dilution recommended by the manufacturer. After washing it in running water, observe presence of residues, and repeat the process if necessary. If any residues is left, remove it manually.

**DISINFECTION:** 10 minutes in 0.2% peracetic acid solution.

**STERILIZATION:** Should be done by autoclaving at a temperature of 121°C up to 135°C (249°F up to 275°F), observing the manufacture's autoclaving instructions manual. Repeat the process for each use.

**STORAGE:** Keep the product in a designated and closed cabinet protected from dust and insects.

**COMPOSITION:** Chromed brass.

**PRECAUTION:** Sterilize the product before use. Do not touch the product at autoclave's inner wall. Do not use steel wool or abrasives to clean the mirror; it may result in scratches and damage the mirrors.

**EXPIRY DATE:** 05 years from manufacturing date in a non-violated package.

**DISPOSAL:** This product disposal should be done according to the current local environmental legislation.

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