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# "Invisible" Clamp

## USE INSTRUCTIONS

Made of thermoplastic and translucent material, it has no radiopacity. It helps to keep the rubber sheet in a stable position next to the teeth and promotes isolation of the operative field. They are indicated for all cases where absolute isolation is necessary, in a practical and safe way. Available in 3 models: GI-01, GI-02 and GI-03.

**VALIDITY:** 05 years from the date of manufacture, in unbroken packaging.

**INSTRUCTIONS FOR USE:** Select the clamp and before using it, clean and sterilize it. Grasp the rubber dam and snap onto the arch for absolute isolation. Take this set to the patient's mouth and mark the position of the tooth to be treated. Using the perforator, puncture the rubber dam at the marked location. Fit the clamp onto the rubber dam and take the entire assembly into the patient's mouth, together with the clamp-holder forceps. Place the clamp on the neck of the tooth and adjust the rubber dam, placing it below the clamp wings. The clamp is an instrument for exclusive use by dentists and must be used according to the professional's scientific knowledge, techniques and preferences.

**CLEANING:** The removal of organic matter must be done in ultrasonic vats using enzymatic soap (minimum 3 enzymes) in the dilution recommended by the manufacturer.

**DISINFECTION:** Use 0.2% peracetic acid solution for 10 minutes.

**STERILIZATION:** The sterilization must be done through an autoclave with a temperature of 121°C to 135°C, observing the recommendations of the autoclave manufacturer's instruction manual.

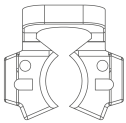
**PACKAGING:** Store the product in an exclusive place, in closed cabinets, protected from dust and insects, away from the sun and rain.

**COMPOSITION:** Thermoplastic resin.


**CAUTION:** Non-sterile product - Sterilize before use. Product indicated for sterilization in autoclave. Manufacturer recommends single use.

**PRODUCT SHELF LIFE:** 05 years from the date of manufacture, in unbroken packaging.

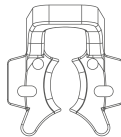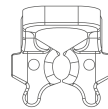
**DISPOSAL:** This product must be disposed of in accordance with current local environmental legislation.

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