Abritec

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Mouth Opener

USE INSTRUCTIONS

It is a silicone device developed to provide comfort to the patient while the professional performs clinical procedures. Available in three sizes and several colors.

COMPOSITION: Silicone and non-toxic pigment.

INDICATIONS: Indicated to help the patient to keep the mouth open in a comfortable way for the professional to perform dental procedures.

INSTRUCTIONS FOR USE: Ask the patient to open the mouth until the necessary opening is obtained. On the opposite side to the one being worked on, place the Abritec between the upper and lower arches, maintaining the patient's mouth opening. The longer side of the trapezium should face anteriorly, providing greater comfort to the patient.

CLEANING: The removal of dirt (organic and inorganic matter) must be done using enzymatic detergent (minimum of three enzymes) with dilution according to the manufacturer's instructions. Cleaning can be done manually or mechanically in an ultrasonic tank. After washing the material, rinse with plenty of water to remove any dirt and/or detergent residue. Carry out the visual inspection and if any remaining dirt is observed, repeat the cleaning process.

DISINFECTION: Keep the product immersed in 0.2% peracetic acid for ten minutes.

STERILIZATION: Dry the product with a clean, dry cloth. Store the material in suitable packaging for sterilization. Sterilization must be carried out by autoclave at a temperature of 121°C to 135°C, in accordance with the autoclave manufacturer's recommendations. The cleaning, disinfection and sterilization process must be carried out after each use.

PACKAGING: Pack the product in an exclusive place, closed cabinets and protected from any contaminants.

WARNING: Select the Abritec size according to the patient's characteristics. Use a dental floss thread through the center hole of the Abritec to hold it like a safety strap.

CAUTION: Sterilize before use. Do not touch the product to the inner walls of the autoclave.

EXPIRY DATE: 05 years from the date of manufacture, in undamaged packaging. The shelf life of this product depends on the care taken during handling.

DISPOSAL: The disposal of this product must be carried out in accordance with current local environmental legislation.

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Rev 01 - 16.01.2023 | Cód 321010880001
ANVISA: 80213420037

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Medical device (Europe).

European Authorized Representative.

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