# Abritec PcD

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# Abritec PwD

## USE INSTRUCTIONS

**ABRITEC PWD:** Mouth opener made of non-toxic and sterilizable material. Equipped with a cable with four different measures to allow proper opening of the mouth. Product indicated for patients with disabilities.

**INDICATIONS:** Indicated to keep the patient's mouth open, facilitating and providing comfort during dental procedures.

**USE INSTRUCCIONES:** Use one side of the cable to support and manipulate Abritec PwD and the other side to insert into the patient's buccal cavity, leaving it stable between the maxilla and the jaw. The opener should be positioned between the premolars and molars in order to keep the mouth open, facilitating buccal hygiene and / or dental procedures. The cable has four different measures to determine the opening of the patient's mouth. Choose one and position it between your teeth. For each side there are two measures, during the procedure it can be changed only with the rotation of the cable.

**CLEANING:** The organic matter removal must be performed in ultrasonic cleaner, using enzymatic soap (minimum 3 enzymes) in dilution recommended by the manufacturer. After washing it in running water, observe presence of residues, and repeat the process if necessary. If it persists, remove residues manually.

**DISINFECTION:** 10 minutes in 0.2% peracetic acid solution.

**STERILIZATION:** After visual inspection check the effectiveness of the cleaning process and pack the product in a sterilization package. The sterilization should be done by autoclaving with a temperature of 121°C to 135°C, observing the recommendations of the autoclave manufacturer's instructions manual.

**STORAGE:** Dispose the product in an exclusive place, in closed cupboards, protected from dust and insects, protected from sun and rain.

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**CONTRAINDICATIONS:** The handling of this product by persons not enable can result in accidents to patients.

**PRECAUTION:** Sterilize before use. Do not touch product to the autoclave inner walls.

**SHELF LIFE:** 05 years from manufacturing date in a non-violated package. Products durability depends on how it is handled. Mishandling it and incorrect sterilization may reduce its useful life. Check the products condition before using it and replace it when observing any damages.

**DISPOSAL:** This product disposal should be done according to the current local environmental legislation.

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