Adaptador para Sugador

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Aspirator adaptor

USE INSTRUCTIONS

INDICATION: Accessory indicated to assist in adapting the suction device to the equipment.

COMPOSITION: Non-toxic silicone.

INSTRUCTIONS FOR USE: Connect the adapter to the equipment (suction hose) and attach the suction device.

CLEANING: Remove organic matter in ultrasonic baths, using enzymatic soap (minimum of three enzymes) in the dilution recommended by the manufacturer. If residues are observed, wash manually under running water. If necessary, repeat the process.

DISINFECTION: Use a 0.2% peracetic acid solution for ten minutes.

STERILIZATION: Dry the product and package it appropriately for sterilization. Sterilize in an autoclave at temperatures between 121°C and 135°C, following the manufacturer's instructions for the autoclave. Repeat the process after each use.

STORAGE: Store the products in their original packaging, sealed, in a clean, dry place away from direct light.

PRECAUTIONS: Sterilize before use. Do not touch the product to the internal walls of the autoclave.

PRODUCT SHELF LIFE: 5 years from the date of manufacture.

DISPOSAL: The disposal of this product must be carried out according to the current local environmental regulations.

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