# Uni – Adesivo Odontológico

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# Uni - Dental Adhesive

## INSTRUCTIONS FOR USE

**COMPOSITION:** Dimethacrylate monomer, Initiators, Adhesive, and Ethyl alcohol.

**INDICATION:** Recommended for promoting adhesion of restorative materials, such as composite resins, to dental structure or previous restorative materials.

### DIRECTIONS FOR USE:

1. Perform conservative cavity preparation as clinically necessary.
2. Condition with 37% phosphoric acid for 30 seconds on enamel and 15 seconds on dentin. After conditioning, rinse thoroughly with abundant water and dry the surface with a gentle stream of air or absorbent cotton pellets or paper.
3. Apply the adhesive actively and evenly using a microbrush on the conditioned surface.
4. Gently air-thin to aid in solvent evaporation.
5. Light-cure the adhesive for 30 seconds.
6. Complete the procedure by applying the desired restorative material over the adhesive.

**DISPOSAL:** Dispose of packaging in accordance with current environmental legislation.

**WARNING:** For use by dental professionals only. Contains ethanol - Flammability risk. Do not apply the product on exposed pulp or in very deep cavities without protection using eugenol-free cavity liners. Do not store or use in conjunction with eugenol-based materials.

**PRECAUTION:** Keep away from sources of light and heat. May polymerize if exposed to light without protection. Close the bottle immediately after use. Use appropriate personal protective equipment (PPE) for both patients and professional staff. Provide protection for mucous membranes in patients sensitive to methacrylates.

**ADVERSE EFFECTS:** Incomplete polymerization or excessive light-curing time may cause mild irritation/sensitivity in some cases.

**STORAGE:** Keep the product in its tightly closed packaging. Protect from direct sunlight. Store the product at temperatures between 5°C to 28°C.

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