# Adfix – Adesivo Ortodôntico

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# Adfix – Orthodontic Adhesive

## INSTRUCTIONS FOR USE

**COMPOSITION:** Methacrylate monomers, aluminum borosilicate glass, sodium fluoride, camphorquinone, and amine.

**INDICATION:** Recommended for bonding metallic, polymeric, and ceramic brackets onto dental surfaces (enamel) for orthodontic treatments.

### DIRECTIONS FOR USE:

#### Placement of orthodontic bracket:

1. Perform dental prophylaxis with oil-free paste.
2. Rinse thoroughly and keep the surface dry.
3. Isolate the field.
4. Condition the enamel surface with phosphoric acid for 30 seconds.
5. Rinse the surface with water and dry it with oil-free air spray (ensure the surface is thoroughly dry).
6. Apply orthodontic adhesive to the bracket base.
7. Position the bracket on the tooth surface, adjusting it correctly, and press firmly to seat it.
8. Remove excess adhesive with an explorer probe, leaving a thin layer of adhesive between the tooth and bracket.
9. Light-cure the adhesive for 80 seconds, 20 seconds on each bracket margin.

#### Removal of orthodontic bracket:

1. After completing orthodontic treatment, remove the bracket using specific pliers.
2. Remove residual orthodontic adhesive with multi-laminated burs.
3. Polish the dental surface.

**DISPOSAL:** Dispose of packaging according to current environmental regulations.

**WARNINGS:** Close the packaging immediately after use. A thick layer of adhesive between the dental surface and orthodontic bracket may compromise adhesion. Installation of the orthodontic archwire can be done immediately after light-curing the last bracket.

**PRECAUTIONS:** Keep away from light and heat sources. May polymerize if exposed to light without protection. Close the packaging immediately after use. Use appropriate PPE for both patients and professional staff.

**ADVERSE EFFECTS:** The product contains methacrylate monomers that may cause allergic reactions or mild irritations in patients sensitive to these substances. Discontinue use in case of allergic reactions.

**STORAGE:** Keep the product in its closed packaging. Protect from direct sunlight. Store the product at temperatures between 5°C and 28°C.

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