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Vertical Lip Retractor

USE INSTRUCTIONS

INDICATIONS: Used to retract the lips in the anterior teeth zone.

USE INSTRUCTIONS: Moisten the patient's lip; Induce the lip frontal retractor for aesthetics in the patient's mouth. First fit one side and then the other in the patient's mouth; Patient has got to close the mouth to keep the product positioned.

CLEANING: The organic matter removal must be made in ultrasonic cleaner, using enzymatic soap (minimum 3 enzymes) in dilution recommended by the manufacturer. After washing it in running water, observe the presence of residues, and repeat the process if necessary. If it persists, remove the residues by manual cleaning.

DISINFECTION: 10 minutes in 0.2 % peracetic acid solution.

STERILIZATION: Immersion in peracetic acid solution 0.2% for 1 hour. After removing the product immersed in the solution, wash it in sterile distilled water and dry it with sterilized gauze. Using PPE and sterile gloves repeat the process each time before use.

STORAGE: Dispose the product in an exclusive place, in closed cupboards, protected from dust and insects, protected from sun and rain.

COMPOSITION: Polyethylene.

PRECAUTION: Sterilize the product before use.

SHELF LIFE: 05 years of manufacture date, in a non violated package.

DISPOSAL: This product disposal should be done according to the current local environmental legislation.

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