Afastadores

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Retractor

- V-SHAPED RETRACTOR
- V2 RETRACTOR
- C-SHAPED RETRACTOR
- MF RETRACTOR
- LIP EXPAND RETRACTOR
- EXPANDEX RETRACTOR

INSTRUCTIONS BY MODEL:

“V-SHAPED RETRACTOR”

AND “V2 RETRACTOR”:

It has round edges providing comfort to the patient and quality oral photographs.

INDICATIONS: Dental photography.

USE INSTRUCTIONS: Take the Retractor to the patient’s mouth, being held by the cable, and can keep it in position by the patient or assistant; pull the product with greater intensity in the side that will be photographed, evidencing the posterior teeth in question. The patient should close the mouth after the retractors have been placed.

COMPOSITION: Polycarbonate.

“C-SHAPED RETRACTOR”:

It has rounded edges providing comfort to the patient and quality oral photographs.

INDICATIONS: Dental photography.

USE INSTRUCTIONS: Take the Retractor by being held by the cable and can keep it in position by the patient or assistant. For front photography keep the Retractors in position (one on the right side and one on the left side) by pulling them with the same intensity to push away the lips. For lateral photographs pull the product with greater intensity on the side that will be photographed, showing the anterior teeth in question. The other Retractor will serve us support, removing the region of the anterior teeth. In both situations the patient should close the mouth after the Retractors have been accommodated.

COMPOSITION: Polycarbonate.

“MF RETRACTOR”:

It has anatomical format that allows perfect fit in the inner part of the upper lip of the patient’s mouth. Due to the greater distance of the lips it puts the teeth in evidence with greater ease, softening the work of the professional. Assists in the separation of the lips, providing quality in the occlusal photographs in the restorative dentist, orthodontics, orthopedics, implantology and general practice.

INDICATIONS: Indicated to separate the lips and provide more quality to intraoral occlusal photographs used in many

dentistry areas such as orthodontics, Functional Jaw Orthopedics (FJO), implantology and general practice.

USE INSTRUCTIONS: Place the MF Retractor in the internal part of the lip (upper or lower) and retract it by pulling it up for maxillary occlusal photographs and down for mandibular occlusal ones.

COMPOSITION: Polycarbonate.

“EXPANDEX RETRACTOR”:

It’s manufactured from a very flexible material allowing perfect fitting in the patient’s mouth exposing the teeth perfectly.

INDICATIONS: It is indicated to separate the lips from the cheek during procedures such as brackets fixation, prophylaxis and anterior intraoral photography.

USE INSTRUCTIONS: Moisten the patient’s lips; bend the Expandex until both ends meet; fit one side after the other. **Note:** The handle should be positioned facing down.

COMPOSITION: Transparent – Polycarbonate. | Other colors – Polyacetal.

“LIP EXPAND RETRACTOR”:

It’s manufactured from flexible material and has an exclusive tongue blocker which provides comfort to the patient during treatments whilst facilitates the procedures. Apart from efficient it also exposes the teeth without having to use the hands to move lips and cheeks apart.

INDICATIONS: For removal of lips and cheek providing work space without the need of the use of professional or auxiliary hands. It facilitates the bonding of brackets, prophylaxis, photography, application of fluoride and aesthetic restorations without any interference of the tongue.

USE INSTRUCTIONS: Moisten the patient’s lips to facilitate adjustment of the Retractor; flex Lip Expand so both extremities meet; first fit one side and then the other side. **Note:** The tongues blocker biggest part should be facing down.

COMPOSITION: Polyacetal.

GENERAL INSTRUCTIONS:

CLEANING: The organic matter removal must be performed in ultrasonic cleaner, using enzymatic soap (minimum 3 enzymes) in dilution recommended by the manufacturer. After washing it in running water, observe presence of residues, and repeat the process if necessary.

DISINFECTION: 10 minutes in 0.2% peracetic acid solution.

STERILIZATION: After visual inspection check the effectiveness of the cleaning process and pack the product in a sterilization package. The sterilization should be done by autoclaving with a temperature of 121°C to 135°C, observing the recommendations of the autoclave manufacturer’s instructions manual. Repeat the process for each use.

STORAGE: Dispose the product in an exclusive place, in closed cupboards, protected from dust and insects, protected from sun and rain.

PRECAUTION: Sterilize the product before use. Do not touch the product at autoclave’s inner wall.

SHELF LIFE: 05 years from manufacturing date in a non-violated package. Products durability depends on how it is handled. Mishandling it and incorrect sterilization may reduce its useful life. Check the products condition before using it and replace it when observing any damages.

DISPOSAL: This product disposal should be done according to the current local environmental legislation.

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Rev 01 - 11.01.2023 | Cód 322010100002

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 Medical device (Europe).

 European Authorized Representative.

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