Óleo Lubrificante Asper

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USE INSTRUCTIONS

LUBRICATING OIL FOR HIGH AND LOW

ROTATION INSTRUMENTS: Provides long life to your dental instruments, as it prevents malfunctions and offers greater protection to internal components.

INDICATIONS: Indicated for lubrication, cleaning and frequent prevention against oxidation and/or drying of high and low rotation dental instruments.

INSTRUCTIONS FOR USE: Shake well before use. Position the instrument to be lubricated horizontally and the oil bottle vertically.

PACKAGING: Store the product in an exclusive place, protected from dust and insects, dry and ventilated, isolated from sources of ignition, oxidants, acids and strong bases.

WARNINGS: Extremely flammable aerosol; In pressurized environments it can break the seal; May be harmful in contact with skin; May be fatal if swallowed and enters airways; Harmful to aquatic organisms.

PRECAUTIONS: Keep away from heat; Spark; open flame; Hot surfaces – Do not smoke; Do not spray over an open flame or other source of ignition; Do not puncture or burn even after use; Avoid release to the environment. Use appropriate PPE when using the product. Do not expose to temperatures above 50°C. In case of fire use CO2 or chemical foam. Avoid contact with eyes. Keep out of the reach of children. Do not reuse the can for other purposes.

FIRST AID: In case of contact with the eyes, wash with plenty of water and seek medical advice. Skin: Wash with running water and soap if irritation persists, seek medical advice. In case of ingestion, do not cause vomiting, seek medical advice and contact the Intoxication/ANVISA Hotline: 0800-722-6001 – EMERGENCY:

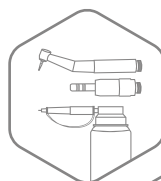
0800 410 148 – 0800 722 6001 – CHEMICAL: 0800 110 8270 – INDUSBELLO: (43) 3378.8000 – 3378.8301. The Chemical Product Safety Information Sheet – FISPQ of this product can be obtained by request through the e-mail indusbello@indusbello.com.br or by phone (43) 3378.8300.

COMPOSITION: Mineral Oil, Propellant butane/propane.

VALIDITY PERIOD OF THE PRODUCT: See the bottom of the bottle.

DISPOSAL: The disposal of this product must be done in accordance with current local environmental legislation.

- **Net Weight 143g (200ml) / 350g (500ml).**
- **Does not contain CFC.**
- **Non-toxic.**
- **Antioxidant and lubricating properties.**
- **Detergent, bactericidal and fungicidal action (except Mentol model).**
- **Cleans, protects and prolongs the life of handpieces.**
- **High and low speed nozzles for precise application.**

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