# Aspirador Endodôntico

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# Endodontic Aspirator

## USE INSTRUCTIONS

**INDICATION:** Irrigating substances aspiration during root canal treatment.

**USE INSTRUCTIONS:** Attach the aspiration needle into the cannula. Attach the cannula into the aspiration hose which is connected to the equipment.

**CLEANING:** The organic matter removal must be performed in ultrasonic cleaner, using enzymatic soap (minimum 3 enzymes) in dilution recommended by the manufacturer. After washing it in running water, observe presence of residues, and repeat the process if necessary.

**DISINFECTION:** 10 minutes in 0.2% peracetic acid solution.

**STERILIZATION:** After visual inspection check the effectiveness of the cleaning process and pack the product in a sterilization package. The sterilization should be done by autoclaving with a temperature of 121°C to 135°C, observing the recommendations of the autoclave manufacturer's instructions manual. Repeat the process with each use.

**STORAGE:** Dispose the product in an exclusive place, in closed cupboards, protected from dust and insects, protected from sun and rain.

**COMPOSITION:** Stainless steel.

**PRECAUTION:** Sterilize the product before use. Do not touch the product at autoclave's inner wall. Products durability depends on how it is handled. Mishandling it and incorrect sterilization may reduce its useful life. Check the products condition before using it and replace it when observing any damages.

**SHELF LIFE:** 05 years from manufacturing date in a non-violated package.

**DISPOSAL:** This product disposal should be done according to the current local environmental legislation.

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