Bandeja

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Tray

USE INSTRUCTIONS

TRAY: Used as an instrument of support in the day to day of professionals of the various areas of dentistry and medicine, as it allows the accommodation and organization of instruments during the performance of procedures. Made of resin that allows sterilization in autoclave. Its format facilitates cleaning and sanitizing. Available in several colors that allow harmony with other products of your clinic.

INDICATIONS: Used in all health and medical areas to accommodate, organize, transport and sterilize products.

USE INSTRUCTIONS: Sterilize the tray in autoclave before the procedures, accommodate the instruments according to your needs.

CLEANING: The organic matter removal must be performed in ultrasonic cleaner, using enzymatic soap (minimum 3 enzymes) in dilution recommended by the manufacturer. After washing it in running water, observe presence of residues, and repeat the process if necessary. If any residues is left, remove it manually.

DISINFECTION: 10 minutes in 0.2% peracetic acid solution.

STERILIZATION: After visual inspection check the effectiveness of the cleaning process and pack the product in a sterilization package. The sterilization should be done by autoclaving with a temperature of 121°C to 135°C, observing the recommendations of the autoclave manufacturer's instructions manual.

STORAGE: Dispose the product in an exclusive place, in closed cupboards, protected from dust and insects, protected from sun and rain.

COMPOSITION: Polymers and non-toxic pigments.

PRECAUTION: Sterilize the product before use. Do not touch the product at autoclave's inner wall.

SHELF LIFE: 05 years from manufacturing date in a non-violated package. Products durability depends on how it is handled. Mishandling it and incorrect sterilization may reduce its useful life. Check the products condition before using it and replace it when observing any damages.

DISPOSAL: This product disposal should be done according to the current local environmental legislation.

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Rev 01 - 17.01.2023 | Cód 323010880001
ANVISA: Dispensado

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 Medical device (Europe).

 European Authorized Representative.

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