Bite Wings

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Bite Wings X-Ray Positioner

USE INSTRUCTIONS

BITE WINGS: Made of sterilizable material, one size fits all.

INDICATIONS: Interproximal radiographs during clinical procedures.

USE INSTRUCTIONS: (For posterior teeth. Not used for anterior teeth): 1. Attach the radiographic film to the Bite Wings flap, making the index parallel to the patient's mouth; 2. Take the set to the patients mouth in the region to be screened; 3. Ask the patient to close his mouth, stabilizing the Bite Wings with his teeth on the bite plate; 4. The Bite Wings indicator rod should pass outside the cone of the x-ray and be parallel to it, and the cone of the x-ray should touch the index of the Bite Wings; 5. In the flap has two markings at its base, the 0/1 indicated for Bambini film and the mark 2 for adult film.

CLEANING: The organic matter removal must be performed in ultrasonic cleaner, using enzymatic soap (minimum three enzymes) in dilution recommended by the manufacturer. After washing it in running water, observe presence of residues, and repeat the process if necessary. If any residues is left, remove it manually.

DISINFECTION: 10 minutes in 0.2% peracetic acid solution.

STERILIZATION: After visual inspection check the effectiveness of the cleaning process and pack the product in a sterilization package. The sterilization should be done by autoclaving with a temperature of 121°C to 135°C, observing the recommendations of the autoclave

manufacturer's instructions manual. Repeat the process with each use.

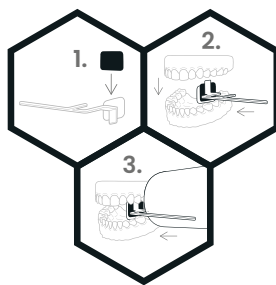
STORAGE: Dispose the product in an exclusive place, in closed cupboards, protected from dust and insects, protected from sun and rain.

COMPOSITION: Polyacetal and non-toxic pigments.

PRECAUTION: Sterilize the product before use. Do not touch the product at autoclave's inner wall.

SHELF LIFE: 05 years from manufacturing date in a non-violated package. Products durability depends on how it is handled. Mishandling it and incorrect sterilization may reduce its useful life. Check the products condition before using it and replace it when observing any damages. **DISPOSAL:** This product disposal should be done according to the current local environmental legislation.

DISPOSAL: The disposal of this product must be carried out in accordance with current local environmental legislation.

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Rev 00 - 18.01.2023 | Cód 327510880016
ANVISA 80213420034

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Medical device
(Europe).

European Authorized
Representative.

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