Cabo para Espelho Cristal

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Crystal Mirror handle

USE INSTRUCTIONS

Used for positioning and handling of crystal mirrors for photography, it offers better grip of the mirrors, dispensing with the contact of the hands with the crystal mirror. The handle has a movable head which provides better mirror positioning. It offers greater security and agility.

COMPOSITION: Aluminium, stainless steel and silicone.

INDICATION: Suitable for fixing and handling crystal mirrors for oral photography.

INSTRUCTIONS FOR USE: Select or mirror Indusbello Glass that best suits the region to be photographed. Fit or espelho escolhido in the cavity of the mobile head of the end, leaving it tight. Lightly or slightly in the patient's mouth, move the head of the end of the mirror to obtain a better position for making the photographic images.

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DISINFECTION: Keep the mirror handle submerged in 0.2% peracetic acid solution for ten minutes.

STERILIZATION: Dry the product and store in suitable packaging for sterilization. Carry out sterilization in an autoclave, at a temperature of 121°C to 135°C, observing the recommendations in the autoclave manufacturer's instruction manual. The cleaning, disinfection and sterilization process must be repeated with each use.

WARNING: Product for professional use. Before use, check the integrity of the product.

CAUTION: Sterilize before use. Do not touch the product on the inner walls of the autoclave.

PACKAGING: Store the product in an exclusive place, in closed cabinets, protected from dust and insects, away from the sun and rain.

PRODUCT SHELF LIFE: 05 years from the date of manufacture, in unbroken packaging. The useful life of this product depends on the care taken during its use.

DISPOSAL: This product must be disposed of in accordance with current local environmental legislation.

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