# Calibrador de Cor

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# Color Calibrator

## INSTRUCTIONS FOR USE

**DESCRIPTION:** 18% gray color calibrator for digital photography.

**COMPOSITION:** Metal and non-toxic silicone.

**INDICATION:** Intended for adjusting the White Balance and Exposure in digital cameras.

**HOW TO USE:** Hold the 18% gray color calibrator in front of the camera so that it completely fills the image frame. Ensure that the calibrator is illuminated by the same light that will illuminate your main subject.

*For "White Balance":*

1. Photograph the calibrator filling the entire image space;
2. Then, use the photo as a new white balance setting in the "White Balance" menu (custom option) or in the image menu (use as a new white balance). Follow the specific instructions for your camera model to manually set the White Balance.

*For "Exposure" adjustment:*

1. When filling the image with the Color Calibrator, access your camera's exposure settings in the "Manual" position and adjust the exposure (shutter, aperture or ISO) as necessary;
2. The adjustments will be correct when the camera's photometry meter is in the "0" position (as shown below). If there is a change in brightness, this process must be repeated.

Note: For exposure adjustment, the use of flash is not recommended.

**CLEANING:** Removal of dirt (organic and inorganic matter) should be done using an enzymatic detergent (at least three enzymes) diluted according to the manufacturer's instructions. Cleaning can be done manually or mechanically in an ultrasonic bath. After washing the material, it should be rinsed with plenty of water to remove any dirt and/or detergent residues. Perform a visual inspection and if any dirt remains, repeat the cleaning process.

**DISINFECTION:** Keep the product immersed in 0.2% peracetic acid for ten minutes.

**STERILIZATION:** Dry the product with a clean, dry cloth. Store the material in suitable packaging for sterilization. Sterilization should be done in an autoclave at a temperature of 121°C to 135°C, according to the manufacturer's recommendations for the autoclave. The process of cleaning, disinfection, and sterilization should be performed after each use.

**DISPOSAL:** In accordance with current local environmental regulations.

**PRECAUTIONS:** Sterilize before use. Do not touch the product to the internal walls of the autoclave. Do not use steel wool sponges or abrasives to clean the contrast.

**STORAGE:** Store the products in their original, sealed packaging, in a clean, dry place away from direct light.

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