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Radiographic positioner made of sterilized material.

INDICATIONS: Indicated for the execution of Periapical radiographs during clinical treatments.

USE INSTRUCTIONS: (For all anterior teeth* upper and lower)

1. Fasten the radiographic film to the pressure flap; 2. Fit the ring and bite device into the positioner; 3. Move the mounted positioner to the patient's mouth so that the tooth to be x-rayed is in the center of the radiographic film; 4. Ask the patient to close their mouth by biting the bite device and stabilizing the positioner; 5. Align the cone of the x-ray with the positioner ring.

PRESENTATION: 1 set for anterior teeth (upper/lower);

1 Bite Block

ATTENTION: Film is not included.

CLEANING: The organic matter removal must be performed in ultrasonic cleaner, using enzymatic soap (minimum three enzymes) in dilution recommended by the manufacturer. After washing it in running water, observe presence of residues, and repeat the process if necessary. If any residues is left, remove it manually.

DISINFECTION: 10 minutes in 0.2% peracetic acid solution.

STERILIZATION: After visual inspection check the effectiveness of the cleaning process and pack the product in a sterilization package. The sterilization should be done by autoclaving with a temperature of 121°C to 135°C, observing the recommendations of the autoclave manufacturer's instructions manual. Repeat the process with each use.

STORAGE: Dispose the product in an exclusive place, in closed cupboards, protected from dust and insects, protected from sun and rain.

COMPOSITION: Polyacetal and Non-toxic Pigments.

PRECAUTION: Sterilize the product before use. Do not touch the product at autoclave's inner wall.

EXPIRY DATE: 05 years from manufacturing date in a non-violated package. Products durability depends on how it is handled. Mishandling it and incorrect sterilization may reduce its useful life. Check the products condition before using it and replace it when observing any damages.

DISPOSAL: This product disposal should be done according to the current local environmental legislation.

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Rev 01 - 18.01.2023 | Cód 327510880020
ANVISA 80213420034

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Medical device
(Europe).

European Authorized
Representative.

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