Cone Indicator – Infantil

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Cone Indicator X-Ray Positioner – Infant

USE INSTRUCTIONS

Set of radiographic positioners used for the acquisition of intraoral dental radiographs. The kit consists of an autoclavable pot containing four radiographic positioners, one for interproximal radiographs of posterior teeth on both sides and three positioners for periapical radiographs: A) for periapical radiographs of the upper posterior teeth on the right side and lower on the left side ; B) for periapical radiographs of the maxillary posterior teeth on the left and mandibular teeth on the right; C) for periapical radiographs of maxillary and mandibular anterior teeth.

COMPOSITION: Polycarbonate, Polyacetal, Silicone and Non-Toxic Pigments.

INDICATION: Indicated for performing periapical and interproximal radiographs.

INSTRUCTIONS FOR USE:

Positioners for periapical radiographs:

Periapical radiographic positioners must be assembled by fitting the guide ring to the positioner shaft and the bite device in the depression next to the film holder. Subsequently, select the radiographic positioner according to the tooth group to be radiographed. Insert the radiographic film between the film holder plates, keeping it in place. Bring the set (positioner with radiographic film) to the patient's mouth, centering it in the region of interest. Ask the patient to occlude in the bite device keeping the set in position. Position the X-ray machine cylinder in line with the positioner guide ring. Carry out the shot on the X-ray machine.

Positioner for interproximal radiographs:

The radiographic positioner is ready to use. Insert the radiographic film between the film support plates, keeping it in position. Bring the set (positioner

with radiographic film) to the patient's mouth, centering it in the region of interest. Ask the patient to occlude the bite wing while holding the assembly in place. Position the X-ray machine cylinder in line with the positioner guide rod. Carry out the shot on the X-ray machine.

CLEANING: The removal of organic matter must be done in ultrasonic vats using enzymatic detergent (minimum 3 enzymes) in the dilution recommended by the manufacturer. Wash in running water. If you notice the presence of residues, repeat the process. If it persists, remove residue by manual cleaning.

DISINFECTION: Afterwards, keep the positioner submerged in 0.2% peracetic acid for ten minutes. Dry the product.

STERILIZATION: Store in suitable packaging for sterilization. Carry out sterilization in an autoclave, at a temperature of 121°C to 135°C, observing the recommendations in the autoclave manufacturer's instruction manual. The cleaning, disinfection and sterilization process must be repeated with each use.

WARNING: Product for professional use. Before use, check the integrity of the product.

CAUTION: Sterilize before use. Do not touch the product on the inner walls of the autoclave.

PACKAGING: Pack the product in an exclusive place, in closed cabinets, protected from dust and insects.

PRODUCT SHELF LIFE: 05 years from the date of manufacture, in unbroken packaging. The useful life of this product depends on the care taken during its use.

DISPOSAL: This product must be disposed of in accordance with current local environmental legislation.

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Rev 02 - 18.01.2023 | Cód 32751088007
ANVISA 80213420034

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Medical device (Europe).

European Authorized Representative.

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