Cone Indicator Digital

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Cone Indicator Digital X-Ray Positioner

USE INSTRUCTIONS

X-RAY POSITIONER: Set of radiographic positioners made of sterilizable material. Positioners allow quick and practical performance of periapical and interproximal radiographs.

INDICATIONS: Indicated for performing intraoral, periapical, and interproximal radiographs to assist in clinical assessment and planning.

COMPOSITION: Non-toxic polyacetal, polycarbonate, pigment and silicone.

INSTRUCTION FOR USE: Identify the region you want to x-ray. Select the fixation device according to your digital sensor model. Attach the fixator to the sensor, in a horizontal position for periapical and interproximal radiographs of the posterior teeth and in a vertical position for periapical radiographs of the anterior teeth. For interproximal radiographs, the sensor must be adjusted in height so that it is centered in relation to the bite wing. To obtain periapical and interproximal radiographs, position the receiver centering the tooth group of the region of interest and ask the patient to occlude in the bite block (for periapical radiographs) or in the bite wing (interproximal radiographs), keeping the positioner stabilized. Align the X-ray machine cylinder with the positioner rod.

THIS PACKAGE CONTAINS: 1 Positioner "A" for posterior teeth (upper left side and lower right side); 1 Positioner "B" for posterior teeth (upper right side and lower left side); 1 "C" positioner for anterior teeth (upper and lower); 3 Bite Devices; 1 Bite Wings (positioner for

interproximal radiography); 4 Rings; 18 radiographic sensor fasteners; 1 Indusbello exclusive autoclavable pot. **ATTENTION:** The sensor is not included with the product.

CLEANING, DISINFECTION AND STERILIZATION: Use enzymatic detergent in ultrasonic vats, as instructed by the manufacturers. Use 0.2% peracetic acid solution for 10 minutes. After visual inspection, pack the product in suitable packaging for sterilization in an autoclave by moist heat, with a temperature between 121°C and 135°C and with a cycle time as directed by the autoclave manufacturer.

PRECAUTIONS: Product suitable for professional use. Sterilize before use.

WARNINGS: Follow the instructions for use. Considering the different models and commercial brands of digital radiographic sensors, among the 18 fixation devices contained in this package, two are expected to be compatible with the dimensions of your digital sensor, one for sensor use in horizontal position and another for sensor use in the horizontal position.

EXPIRING DATE: 05 years from manufacturing date in a non-violated package.

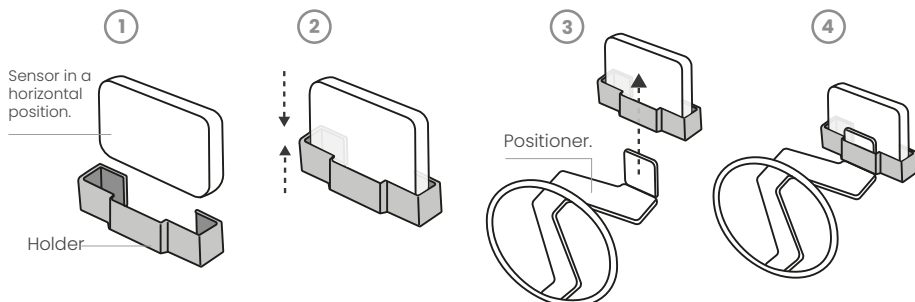
STORAGE: Keep the product in a designated and closed cabinet protected from dust and insects.

DISPOSAL: This product disposal should be done according to the current local environmental legislation.

INSTRUCTIONS FOR USE - DIGITAL SENSOR HOLDERS


SENSOR IN HORIZONTAL POSITION:

- 1 - Check which holder has the **proper dimensions** for the sensor.
- 2 - Fit the holder to the digital sensor in a **horizontal position**.
- 3 and 4 - **Fit the set** (holder + sensor) on the radiographic positioner.

SENSOR IN VERTICAL POSITION:

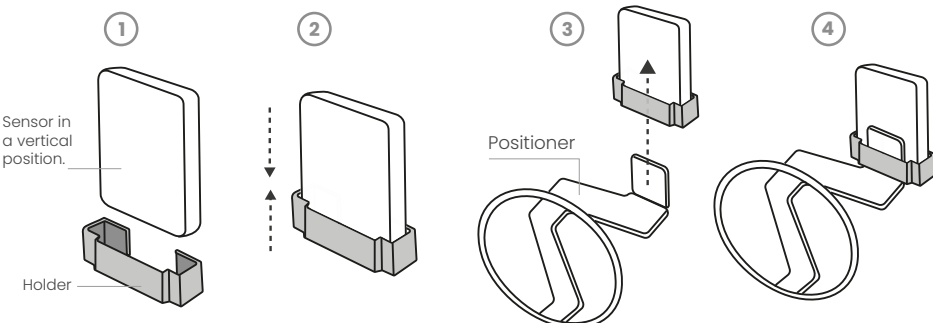- 1 - Check which holder has the **proper dimensions** for the sensor.
- 2 - Fit the holder to the digital sensor in a **vertical position**.
- 3 and 4 - **Fit the set** (holder + sensor) on the radiographic positioner.

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Rev 01 - 18.01.2023 | Cód 327510880010
ANVISA: 80213420034

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Medical device (Europe).

European Authorized Representative.

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