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Digital Sensor Holder

Set of 18 digital sensors holders of different sizes and for use by different brands.

INDICATION OF USE/PURPOSE: Device for hold the digital sensor to the Indusbello radiographic positioner. Due to the different dimensions of the radiographic sensors of different brands/commercial models, 18 fixation units are available, being nine units for use with the digital sensor in the horizontal position and nine units for use with the digital sensor in the vertical position.

DIRECTIONS FOR USE: Identify the fixation device that best fits your digital sensor measurements. Attach the device to the sensor and attach it to the Indusbello radiographic positioner. For each digital sensor, two fixation units will be needed: one unit for accommodating the digital radiographic sensor in the vertical position (for anterior teeth) and one unit for accommodating the digital radiographic sensor in the horizontal position (for posterior teeth). It is not necessary to use the other fixation units.

COMPOSITION: Polyacetal

STERILIZATION: Product sterilizable in an autoclave, with a temperature between 121°C and 135°C, observing the recommendations in the manufacturer's instruction manual. Repeat the process for each use. Autoclaving time: one cycle as directed by the autoclave manufacturer.

STORAGE: Store the product in an exclusive place for health products, in closed cabinets, protected from dust and insects.

CLEANING: The removal of organic matter must be done in ultrasonic vats using enzymatic soap (minimum 3 enzymes) in dilution recommended by the manufacturer. Rinse under running water and if the presence of residues is observed, repeat the process.

DISINFECTION: Use 0.2% peracetic acid solution for 10 minutes.

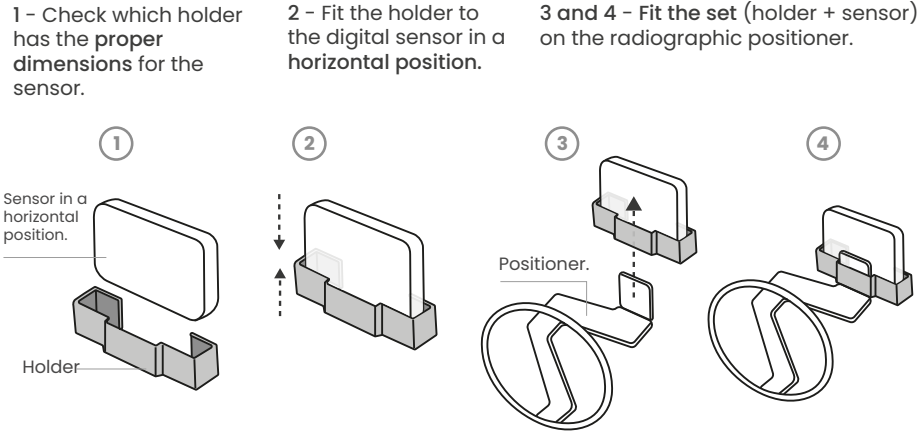
VALIDITY: 5 years.

DISPOSAL: The disposal of this product must be done in accordance with current local environmental legislation.


PRECAUTIONS: Read the instructions for use before using the product. This Kit contains 18 (eighteen) holder devices with different dimensions to adapt to the different digital sensors available on the market. The expectation is that up to two fasteners are compatible with the sensor, a fastener to attach the digital sensor in a vertical position and other fasteners to attach the digital sensor in a horizontal position. Product intended for professionals in the dental field. Product supplied non-sterile. Sterilization in an autoclave before use is recommended. Do not touch the product to the autoclave walls. Keep the product lot number.

INSTRUCTIONS FOR USE - DIGITAL SENSOR HOLDERS

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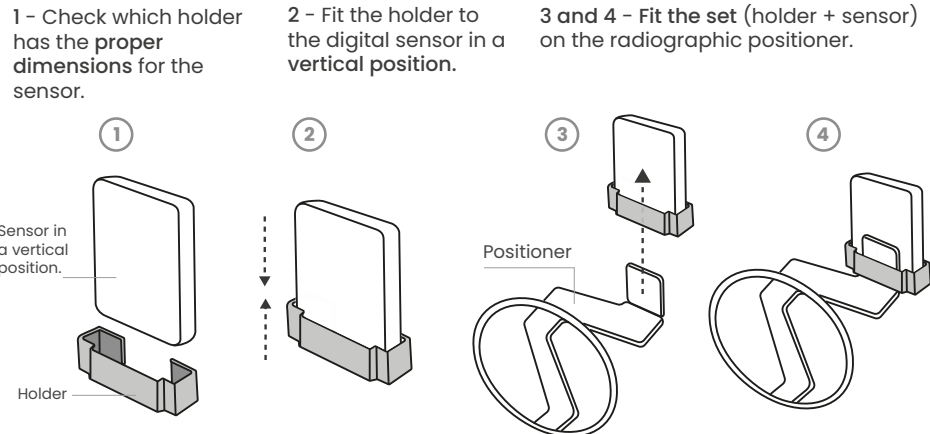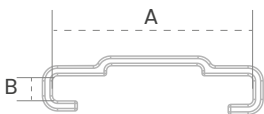
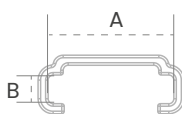
Rev 02 - 18.01.2023 | Cód 307510880001
ANVISA: 80213420034

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Medical device (Europe).

European Authorized Representative.

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