Contraste

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Contrast

USE INSTRUCTIONS

CONTRAST: Product developed in aluminum and with non-stick black non-stick coating. Allows evidence of teeth for better photographic performance. Its format ensures greater comfort and safety at the time of use, as one side is used as contrast and the other as a handle.

COMPOSITION: Aluminum with non-toxic non-stick coating.

INDICATIONS: Indicated as a background to contrast with the teeth in intraoral photographs.

No. 04, 05 and 07: Front/anterior photos.

No. 08: Side/back photos.

Occlusal: Adult occlusal photo.

Children's occlusal: Children's occlusal photo.

INSTRUCTION FOR USE: Select the contrast according to the region to be photographed. The contrast must be placed in the mouth, behind the teeth in the region of interest, and held in position by the patient or assistant.

CLEANING: The removal of dirt (organic and inorganic matter) must be done using enzymatic detergent (minimum of three enzymes) with dilution according to the manufacturer's instructions. Cleaning can be done manually or mechanically in an ultrasonic tank. After washing the material, rinse with plenty of water to remove any dirt and/or detergent

residue. Carry out the visual inspection and if any remaining dirt is observed, repeat the cleaning process.

DISINFECTION: Keep the product immersed in 0.2% peracetic acid for ten minutes.

STERILIZATION: Dry the product with a clean, dry cloth. Store the material in suitable packaging for sterilization. Sterilization must be carried out by autoclave at a temperature of 121°C to 135°C, in accordance with the autoclave manufacturer's recommendations. The cleaning, disinfection and sterilization process must be carried out after each use.

WARNING: Sterilize before use. Do not use steel wool sponges or abrasives to clean the contrast.

CAUTION: Sterilize before use. Do not touch the product to the inner walls of the autoclave.

PACKAGING: Pack the product in an exclusive place, closed cabinets and protected from any contaminants.

EXPIRY DATE: 05 years from the date of manufacture, in undamaged packaging. The shelf life of this product depends on the care taken during handling.

DISPOSAL: The disposal of this product must be carried out in accordance with current local environmental legislation.

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Rev 02 - 12.01.2023 | Cód 326210880002
ANVISA: 80213420023

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Medical device (Europe).

European Authorized Representative.

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