Pote Dappen Dedal

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Finger Holder Dappen Dish

USE INSTRUCTIONS

The Finger Holder Dappen Dish is made of stainless steel and has a rod that can be adjusted to fit the professional's finger, facilitating clinical work.

COMPOSITION: Stainless steel.

INDICATIONS: Indicated for handling and packaging products for dental application such as prophylactic paste and acrylic resin, for example.

INSTRUCTIONS FOR USE: According to the professional's needs, place the dental materials in the dappen and fit the stem of the dappen to the professional's finger. To avoid hardening of the handled materials, clean the dappen immediately after use.

CLEANING: The removal of dirt (organic and inorganic matter) must be done using enzymatic detergent (minimum of three enzymes) with dilution according to the manufacturer's instructions. Cleaning can be done manually or mechanically in an ultrasonic tank. After washing the material, rinse with plenty of water to remove any dirt and/or detergent residue. Carry out the visual inspection and if any remaining dirt is observed, repeat the cleaning process.

DISINFECTION: Keep the product immersed in 0.2% peracetic acid for ten minutes.

STERILIZATION: Dry the product with a clean, dry cloth. Store the material in suitable packaging for sterilization. Sterilization must be carried out by autoclave at a temperature of 121°C to 135°C, in accordance with the autoclave manufacturer's recommendations. The cleaning, disinfection and sterilization process must be carried out after each use.

CAUTION: Clean the Finger Holder Dappen Dish immediately after use. Sterilize before use. Do not touch the product to the inner walls of the autoclave.

PACKAGING: Pack the product in an exclusive place, closed cabinets and protected from any contaminants.

EXPIRY DATE: 05 years from the date of manufacture, in undamaged packaging. The shelf life of this product depends on the care taken during handling.

DISPOSAL: The disposal of this product must be carried out in accordance with current local environmental legislation.

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