Pote Dappen Silicone

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Silicone Dappen Dish

USE INSTRUCTIONS

The Silicone Dappen Dish has a concavity that can be used according to the professional's needs. It is unbreakable, non-stick, does not deform and has a base that attaches easily to the clinical table, facilitating the professional's work. Made of silicone and available in different colors.

COMPOSITION: Non-toxic silicone and pigments.

INDICATIONS: Indicated for handling and packaging products for dental application such as prophylactic paste and acrylic resin, for example.

INSTRUCTIONS FOR USE: Place the dental material in the concavity of the Silicone dappen. To avoid hardening of the handled materials, clean the dappen immediately after use.

CLEANING: The removal of dirt (organic and inorganic matter) must be done using enzymatic detergent (minimum of three enzymes) with dilution according to the manufacturer's instructions. Cleaning can be done manually or mechanically in an ultrasonic tank. After washing the material, rinse with plenty of water to remove any dirt and/or detergent residue. Carry out the visual inspection and if any remaining dirt is observed, repeat the cleaning process.

DISINFECTION: Keep the product immersed in 0.2% peracetic acid for ten minutes.

STERILIZATION: Dry the product with a clean, dry cloth. Store the material in suitable packaging for sterilization. Sterilization must be carried out by autoclave at a temperature of 121°C to 135°C, in accordance with the autoclave manufacturer's recommendations. The cleaning, disinfection and sterilization process must be carried out after each use.

CAUTION: Clean the silicone dappen immediately after use. Sterilize before use. Do not touch the product to the inner walls of the autoclave.

PACKAGING: Pack the product in an exclusive place, closed cabinets and protected from any contaminants.

EXPIRY DATE: 05 years from the date of manufacture, in undamaged packaging. The shelf life of this product depends on the care taken during handling.

DISPOSAL: The disposal of this product must be carried out in accordance with current local environmental legislation.

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