Pote Dappen Duo

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Duo Dappen Dish

USE INSTRUCTIONS

The Duo Dappen Dish is the only dappen pot that has four concavities that can be used according to professional needs. Manufactured in polymer and available in several cores.

COMPOSITION: Polycarbonate and non-toxic pigments.

INDICATIONS: Indicated for handling and storing products for dental application, such as prophylactic paste, for example.

INSTRUCTIONS FOR USE: As required by the professional, select the cavity to be used. Place the dental material in the concavity of the Dappen Duo pot. To avoid hardening of the materials handled, clean the dappen pot immediately after use.

CLEANING: The cleaning, disinfection and sterilization processes must be done with the Dappen Duo pot disassembled. To disassemble, turn the parts away from each other with both hands. The removal of dirt (organic and inorganic materials) must be done with the use of enzymatic detergent (minimum of three enzymes) with dilution according to the manufacturer's instructions. Cleaning can be done manually or mechanically in an ultrasonic vat. After washing the material, rinse it with plenty of water to remove any dirt and/or detergent residue. Carry out the visual inspection and if any remaining dirt is observed, repeat the cleaning process.

DISINFECTION: Keep the product immersed in 0.2% peracetic acid for ten minutes.

STERILIZATION: Dry the product with a clean, dry cloth. Store material in suitable packaging for sterilization. Sterilization must be done by autoclave at a temperature of 121°C to 135°C, according to the autoclave manufacturer's recommendations. The cleaning, disinfection and sterilization process must be carried out after each use.

WARNING: Do not use or handle acrylic resin in the Dappen Duo pot. The cleaning, disinfection and sterilization processes must be carried out with the Dappen Duo pot disassembled.

CAUTION: Clean the dappen Duo pot immediately after use. Sterilize before using. Do not touch the product on the inner walls of the autoclave.

PACKAGING: Pack the product in an exclusive place, closed cabinets and protected from any contaminants.

SHELF LIFE: 05 years from the date of manufacture, in unbroken packaging. The useful life of this product depends on the care taken during its handling.

DISPOSAL: This product must be disposed of in accordance with current local environmental legislation.

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