# Dischi – Discos de Polimento

## Centro Metálico e Centro de Silicone

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Rev 00 - 27.11.2023 | Cód 321510880016  
ANVISA: 80213420068

# Dischi – Polishing Disks

## *Metal connector and Silicon connector*

### INSTRUCTIONS FOR USE:

**DESCRIPTION:** Polyester disc with abrasive in four grits: coarse (purple disc), medium (blue disc), fine (green disc), and extra-fine (pink disc). The discs are available in two sizes: 12 mm and 14 mm in diameter. There are two attachment systems on the disc on the mandrel: pop-on (metal center) or snap-on (silicone center).

**COMPOSITION:** Polishing discs: polyester with alumina abrasives (aluminum oxide); Mandrel: stainless steel.

**INDICATION:** The polishing disc is indicated for finishing and polishing composite resin restorations.

**INSTRUCTIONS FOR USE:** Attach the polishing disc to the mandrel, and this assembly should be fitted into a low-speed handpiece with a speed of up to 8,000 rpm. The polishing disc should be used with gentle manual pressure on the surface to be polished. The disc should be used in sequence according to the grit (from the highest to the lowest grit): purple, blue, green, and pink.

**HANDLING CONDITIONS:** The manufacturer recommends single-use for the polishing disc and it should not be sterilized. Ready for use. The mandrel should undergo cleaning, disinfection, and sterilization processes after each use.

**CLEANING:** Dirt on the mandrel should be removed using enzymatic detergent (minimum of three enzymes) with dilution as per the manufacturer's instructions. Cleaning can be done manually or mechanically in an ultrasonic bath. After washing the mandrel, rinse it thoroughly with abundant water to

remove any dirt and/or detergent residues. Perform visual inspection, and if any dirt residue is observed, repeat the cleaning process.

**DISINFECTION:** Keep the mandrel immersed in 0.2% peracetic acid for ten minutes.

**STERILIZATION:** Dry the mandrel with a clean, dry cloth. Store the material in suitable packaging for sterilization. Sterilization should be done by autoclave at a temperature of 121°C to 135°C, according to the autoclave manufacturer's recommendations.

**DISPOSAL:** Discard polishing discs and mandrels in accordance with local environmental regulations.

**WARNINGS:** The manufacturer recommends single-use for polishing discs. The mandrel should be sterilized before use. The disc should be used at a speed of up to 8,000 rpm. It is necessary to follow the sequence of disc use: purple, blue, green, and pink. Product for professional use.

**PRECAUTIONS:** The professional should wear personal protective equipment (PPE), and the patient should wear protective glasses during product use. Before activating low speed, ensure that the polishing disc is properly fitted in the contra-angle. Do not touch the mandrel to the inner walls of the autoclave.

**STORAGE:** Store and transport the products in their original, closed packaging in a clean, dry place, protected from direct light.

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