Dispositivo ClarkTec

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FDA Food and Drug Administration (USA).

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ClarkTec Device

Compatible with *Indusbello* digital and analog X-Ray positioners

INSTRUCTIONS FOR USE

DESCRIPTION: Recommended to be adapted to *Indusbello* radiographic positioners to obtain periapical radiographs in orthoradial, mesioradial and distoradial views. Allows you to perform the Clark technique quickly and practically. Product designed in partnership with dental surgeon Andre Caliman.

COMPOSITION: Polyacetal.

INDICATION: Indicated to assist in the execution of periapical radiographs with the Clark technique (mesialization and distalization of the X-ray beam).

HOW TO USE: This device is an accessory that must be used on *Indusbello* radiographic positioners. Assemble the radiographic positioner by fitting the ClarkTec device between the rod and the guidance ring. Adjust the ClarkTec device according to the desired incidence (orthoradial, mesioradial or dystoradial). Proceed with the radiographic technique according to the tooth group of interest.

CLEANING: Use enzymatic detergent in ultrasonic tanks, according to manufacturers' instructions.

DISINFECTION: keep the radiographic positioner immersed in a 0.2% peracetic acid solution for 10 minutes. Perform visual inspection, dry the product with a clean cloth.

STERILIZATION: Pack the positioner in packaging suitable for sterilization in an autoclave at a temperature between 121°C and 135°C, as recommended by the autoclave manufacturer. The cleaning, disinfection and sterilization process must be carried out after each use.

DISPOSAL: The product must be disposed of in accordance with current local environmental legislation.

WARNINGS: Handle the device with care when moving the axis to the right or left.

PRECAUTIONS: Product indicated for professional use. Sterilize before use. Do not touch the product to the internal walls of the autoclave.

STORAGE: Store the product in its original, closed packaging, in a clean, dry place and away from direct light.


EXPIRY DATE: 05 years from the date of manufacture, in undamaged packaging. The useful life of this product depends on the care taken when handling it.

ILLUSTRATIONS OF USE:

ASSEMBLY OF THE CLARKTEC DEVICE:


STEP 1:

Fit the ClarkTec device (2) next to the positioner body (3), in the position indicated below.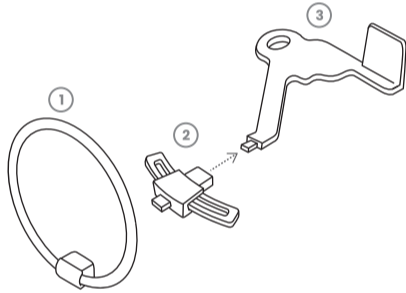


STEP 2:

Attach the orientation ring (1) to the other end of the ClarkTec device (2).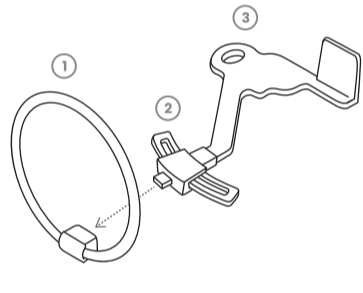


USE IN ORTHORADIAL INCIDENCE

Arrows on the device are aligned.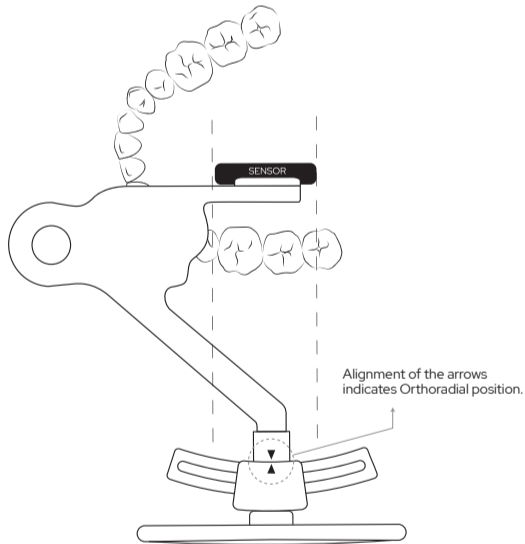

USE IN MESIORADIAL OR DYSTORADIAL INCIDENCE

The mesioradial or dystoradial incidence depends on the side of the dental arch.

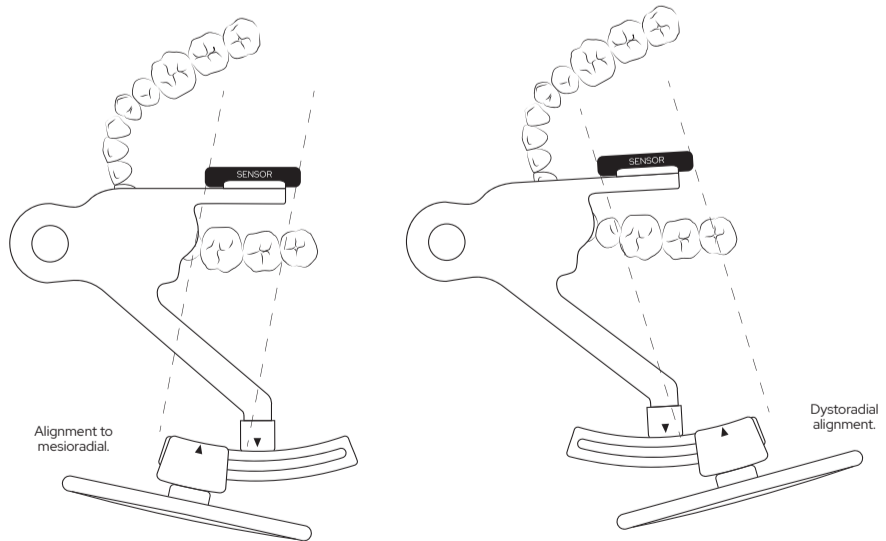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