Dispositivo para Mordida

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Bite Device

USE INSTRUCTIONS

BITE DEVICE: Ideal device for use in the Indusbello Cone Indicator in conventional and digital models, both in autoclavable and chemical sterilization versions.

COMPOSITION: Non-toxic silicone.

INDICATIONS: Indicated for the patient to occlude when taking periapical radiographs with the use of the radiographic positioner in order to stabilize the radiographic positioner in the mouth.

INSTRUCTIONS FOR USE: Attach the bite device to the Indusbello radiographic positioner in the bite device housing adjacent to the image receptor holder. After placing the radiographic positioner according to the tooth group to be radiographed, ask the patient to occlude the bite device in order to stabilize the positioner in the mouth.

CLEANING: The removal of dirt (organic and inorganic materials) must be done using an enzymatic detergent (minimum of three enzymes) with dilution according to the manufacturer's instructions. Cleaning can be done manually or mechanically in an ultrasonic vat. After washing the material, rinse it with plenty of water to remove any dirt and/or detergent residue. Carry out the visual inspection and if any remaining dirt is observed, repeat the cleaning process.

DISINFECTION: Keep the product immersed in 0.2% peracetic acid for ten minutes.

STERILIZATION: Dry the product with a clean, dry cloth. Store material in suitable packaging for sterilization. Sterilization must be done by autoclave at a temperature of 121°C to 135°C, according to the autoclave manufacturer's recommendations. The cleaning, disinfection and sterilization process must be carried out after each use.

PACKAGING: Pack the product in an exclusive place, closed cabinets and protected from any contaminants.

WARNING: Product for exclusive use in Indusbello radiographic positioner.

CAUTION: Sterilize before use. Do not place the product against the walls of the autoclave.

SHELF LIFE OF THE PRODUCT: 05 years from the date of manufacture, in unbroken packaging. The lifespan of this product depends on the

DISPOSAL: This product must be disposed of in accordance with current local environmental legislation.

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Rev 01 - 18.01.2023 | Cód 327510880018
ANVISA 80213420034

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Medical device (Europe).

European Authorized Representative.

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