E-Mold

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E-Mold

USE INSTRUCTIONS

E-MOLD is an elastic transparent highly durable square shaped plate for custom fitted trays making, its material doesn't irritate the gums enabling adaptation to the model with no voids at three different thickness allowing accuracy and minimizing impression material distortion.

INDICATIONS: To make custom fitted trays through vacuum forming system. It can be used to make whitening trays, mouthguards and occlusal splints for the treatment of sleep bruxism.

USE INSTRUCTIONS: 1. Using molding material, mold the dental arcade and fill it with plaster. 2. Using the plaster model acquired, apply to the vestibular teeth face a uniform layer between 0.5 to 1.0 mm thick in photopolymerizable resin or acrylic resin self-polymerizable, forming an empty space. In this procedure, distance from the gum between 1-2 mm including the papilla should be maintained. 3. Fix the plaque in the Vacuum Forming machine centralizing the plaster model under the plaque and turn the machine on. When the plaque

get soft and a slight 2 to 3 cm curve, lower it on the plaster model and activate the vacuum according to the equipment usage instructions. The plaque should fit the model reaching the gums edges. 4. Wait for the custom fitted tray to cool down, cut it around 1 cm from the molded part. 5. Accommodate the custom fitted tray on the plaster model until handing it in to the patient. 6. It is recommended to store the custom fitted tray in the **Indusbello's BOX**.

PRECAUTION: Do not handle the plate without hands protection as the material could be hot after vacuum forming process.

COMPOSITION: Copolymer of ethylene and 100% vinyl acetate (EVA).

EXPIRY DATE: 02 years from manufacturing date in a non-violated package.

DISPOSAL: This product disposal should be done according to the current local environmental legislation.

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