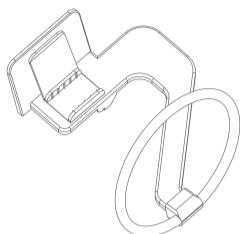
Posicionador Endo Cone Duo

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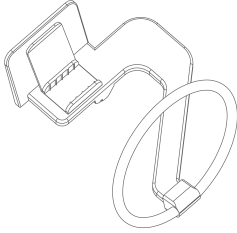
Endo Cone Duo X-Ray Positioner


INSTRUCTIONS FOR USE

CONTAINS: 1 autoclavable jar containing 6 positioners, including 1 positioner for interproximal radiography and 5 positioners for periapical radiographs in five models, three for periapical radiographs (**Models 1, 2, and 3**) and two for obtaining periapical radiographs during endodontic treatment (**Models A and B**):


- **Model 1:** for periapical radiographs of upper right posterior teeth and lower left posterior teeth;

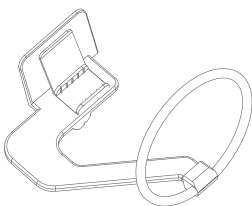
- **Model 2:** for periapical radiographs of upper left posterior teeth and lower right posterior teeth:

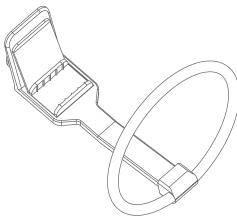
- **Model 3:** for periapical radiographs of upper and lower anterior teeth:

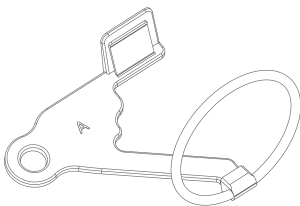
- **Model A:** for periapical radiographs of anterior teeth and for periapical radiographs of upper left posterior teeth and lower right posterior teeth:

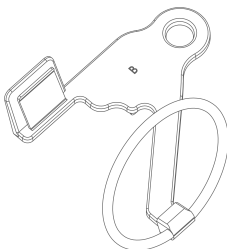- **Model B:** for periapical radiographs of anterior teeth and for periapical radiographs of upper right posterior teeth and lower left posterior teeth:

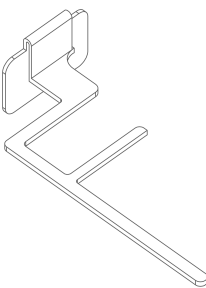- **Model Interproximal:** for interproximal radiographs of posterior teeth on both sides.

Models A and B positioners feature a design that allows periapical radiographs to be taken even when the patient is under absolute isolation during endodontic treatment. Additionally, these positioners come with exclusive bite devices with heights of 14 mm and 16 mm to be used according to the professional's needs.

COMPOSITION: Polycarbonate, polyacetal, silicone, and non-toxic pigments.

INDICATION: Recommended for obtaining periapical radiographs, even in endodontic treatment situations, and for obtaining interproximal radiographs.

INSTRUCTIONS FOR USE:

For obtaining periapical radiographs (Models 1, 2, and 3 positioners):

1. Select the radiographic positioner according to the dental group to be radiographed;
2. Assemble the radiographic positioner by attaching the guidance ring to the shaft and the bite device near the film support;
3. Insert the radiographic film into the film support;
4. Place the entire assembly (positioner with the radiographic film) into the patient's mouth, positioning the film so that it is centered on the dental group to be radiographed;
5. Ask the patient to bite the bite device to keep the positioner stable.
6. Align the X-ray machine cylinder with the guidance ring of the radiographic positioner.

For obtaining interproximal radiographs:

1. Select the radiographic positioner for interproximal technique;
2. Insert the radiographic film into the film support;
3. Place the entire assembly (positioner with the radiographic film) into the patient's mouth, positioning the film so that it is centered on the dental group to be radiographed;
4. Ask the patient to bite to keep the positioner stable;
5. Align the X-ray machine cylinder with the guidance shaft of the radiographic positioner.

To perform periapical radiographs during endodontic treatment (Models A and B positioners):

1. Select the positioner piece according to the area to be radiographed;
2. Assemble the radiographic positioner by attaching the guidance ring to the shaft and the bite device near the film support;
3. Insert the radiographic film into the film support;
4. Place the entire assembly (positioner with the radiographic film) into the patient's mouth, positioning the film so that it is centered on the dental group to be radiographed;
5. Ask the patient to bite the bite device to keep the positioner stable;
6. Align the X-ray machine cylinder with the guidance ring of the Endo RH Plus positioner.

For obtaining periapical radiographs of anterior teeth, the Endo RH bite device can be removed; in this case, the patient should keep their mouth open, and the positioner should be kept stable by the shaft.

CLEANING: Organic matter removal should be done in ultrasonic tanks using enzymatic detergent (minimum 3 enzymes) at the dilution recommended by the manufacturer. Rinse with running water. If residues are observed, repeat the process. If residues persist, remove them through manual cleaning.

DISINFECTION: Subsequently, keep the positioner submerged in 0.2% peracetic acid for ten minutes. Dry the product.

STERILIZATION: Store in suitable packaging for sterilization. Perform sterilization in an autoclave, at a temperature of 121°C to 135°C, observing the recommendations of the autoclave manufacturer's instructions manual. The cleaning, disinfection, and sterilization process should be repeated for each use.

DISPOSAL: Disposal of this product should be done according to local environmental legislation.

WARNING: For professional use only. Check the integrity of the product before use.

PRECAUTION: Product intended for professional use. Sterilize before use. Do not touch the product to the inner walls of the autoclave.

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