Endofit

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Endofit

USE INSTRUCTIONS

ENDOFIT: Is a milimeter scale endodontic ruler manufactured from high performance polymer. It is light, autoclavable, ambidextrous and available in many colors with an exclusive stopper and guiding hole to adjust the file perfectly and precisely to the tooth.

INDICATIONS: It is used in endodontic odontology to avoid mistakes in interpretation or measurement which may compromise the success of endodontic treatments.

USE INSTRUCTIONS: Adjust the endodontic file according to the tooth canal's working length (odontometry). To have better results use the guiding hole and adjust the file precisely.

CLEANING: The organic matter removal must be performed in ultrasonic cleaner, using enzymatic soap (minimum 3 enzymes) in dilution recommended by the manufacturer. After washing it in running water, observe presence of residues, and repeat the process if necessary. If it persists, remove residues manually.

DISINFECTION: 10 minutes in 0.2% peracetic acid solution.

STERILIZATION: After visual inspection check the effectiveness of the cleaning process and pack the product in a sterilization package. The sterilization should be done by autoclaving with a temperature of 121°C to 135°C, observing the recommendations of the autoclave manufacturer's instructions manual.

STORAGE: Dispose the product in an exclusive place, in closed cupboards, protected from dust and insects, protected from sun and rain.

COMPOSITION: Thermoplastic resin and pigment.

PRECAUTION: Sterilize the product before use. Do not touch the product at autoclave's inner wall.

SHELF LIFE: 05 years from manufacturing date in a non-violated package. Products durability depends on how it is handled. Mishandling it and incorrect sterilization may reduce its useful life. Check the products condition before using it and replace it when observing any damages.

DISPOSAL: This product disposal should be done according to the current local environmental legislation.

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