Endo RH Plus

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Endo RH Plus X-Ray Positioner

USE INSTRUCTIONS

ENDO RH PLUS: The set of radiographic positioners is manufactured in sterilizable material in adult size and is commercialized with mounted parts that streamline the professional's work. The sets of positioners allow transoperative radiographs under absolute isolation in endodontics (conometry and odontometry) and implantology. **Note:** The Endo RH Plus positioner set comes with exclusive bite devices that are used to compensate for the leftovers of the file and the cone, respectively in odontometry and conometry, so there are 2 sizes: 14mm or 16mm being used according to the needs of the professional.

INDICATIONS: Indicated for the execution of transoperative radiographs in endodontics and implantology. For endodontics, it is used for radiographic shots under absolute isolation, that is, transoperative radiographs during canal treatment (odontometry and conometry) and for implantology: in the verification of the paralleliser / direction indicator in relation to the maxillary sinus, nasal cavities and mentonian nerve.

CLEANING: The organic matter removal must be performed in ultrasonic cleaner, using enzymatic soap (minimum three enzymes) in dilution recommended by the manufacturer. After washing it in running water observe presence of residues and repeat the process if necessary. If any residue is left, remove it manually.

DISINFECTION: 10 minutes in 0.2% peracetic acid solution.

STERILIZATION: After visual inspection check the effectiveness of the cleaning process and pack the product in a sterilization package. The sterilization should be done by autoclaving with a temperature of 121°C to 135°C, observing the recommendations of the autoclave manufacturer's instructions manual. Repeat the process with each use.

PRESENTATION: 1 SET A for Left maxillary molars, premolars and central incisors & Right mandibular molars, premolars and central incisors; **1 SET B** Right maxillary molars and Left mandibular molars & Maxillary central incisor and mandibular central incisor

2 Bite Devices (14mm)

2 Bite Devices (16mm)

1 Autoclavable Case (Indusbello exclusive)

ATTENTION: Film is not included.

USE INSTRUCTIONS:

• **Set A** (for the posterior (upper left and lower right) and anterior (upper and lower) teeth);

• **Set B** (for the posterior (upper right and lower left) and anterior (upper and lower) teeth);

Posterior teeth: 1. Attach the radiographic film to the pressure flap (the film should be placed horizontally). **2.** Attach the ring and bite device to the positioner. **3.** Take the set to the patients mouth so that the tooth to be radiograph is in the center of the radiographic film. **4.** Ask the patient to close their mouth and bite the bite device stabilizing automatically the positioner. **5.** Align the cone to the x-ray with the Endo RH Plus positioner ring.

Anterior teeth: Repeat the above procedures, but the radiographic film should be placed vertically. In the case of anterior Teeth, the bite device will be dispensed, the patient should keep his mouth open and stabilize the positioner by holding it by the stem (with thumb and forefinger)

In endodontics: 1. Loosen the rubber sheet on the opposite side of the tooth to be x-rayed. **2.** If it is necessary to dissociate roots, adjust the cone of the x-ray to perform a mesio-radial and distal-radial radiograph.

COMPOSITION: Polyacetal, polycarbonate, nontoxic silicone and pigment.

STORAGE: Dispose the product in an exclusive place, in closed cupboards, protected from dust and insects, protected from sun and rain.

PRECAUTION: Sterilize the product before use. Do not touch the product at autoclave's inner wall.

EXPIRY DATE: 05 years from manufacturing date in a non-violated package. Products durability depends on how it is handled. Mishandling it and incorrect sterilization may reduce its useful life. Check the products condition before using it and replace it when observing any damages.

DISPOSAL: This product disposal should be done according to the current local environmental legislation.

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Rev 01 - 18.01.2023 | Cód 327510880014
ANVISA 80213420034

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 Medical device (Europe).

 European Authorized Representative.

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