Espátula Plástica

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Plastic Spatula

USE INSTRUCTIONS

Made of thermoplastic resins, in many colors. It has a smooth surface, suitable for manipulating materials. It can be used in dentistry to facilitate alginate or plaster homogenization while making study models, prosthetic moldings and orthodontic documentation. It can also be used in other fields such as cosmetic, veterinary and laboratory.

INDICATIONS: For general use. The plastic spatula is suitable for materials homogenization.

USE INSTRUCTIONS: Use the Plastic Spatula together with the Indusbello's Bowl. Hold it firmly and mix the material according to the manufacturer's use instructions.

CLEANING: The organic matter removal must be performed in ultrasonic cleaner using enzymatic soap (minimum 3enzymes) in dilution recommended by the manufacturer. After washing it in running water, observe presence of residues, and repeat the process if necessary. If any residues is left, remove it manually.

DISINFECTION: 10 minutes in 0.2% peracetic acid solution.

STERILIZATION: After visual inspection check the effectiveness of the cleaning process and pack the product in a sterilization package. The sterilization should be done by autoclaving with a temperature of 121°C to 135°C, observing the recommendations of the autoclave manufacturer's instructions manual.

STORAGE: Dispose the product in an exclusive place, in closed cupboards, protected from dust and insects, protected from sun and rain.

COMPOSITION: Polypropylene and pigments.

PRECAUTION: Sterilize the product before use. Do not touch the product at autoclave's inner wall.

SHELF LIFE: 05 years from manufacturing date in a non-violated package. Products durability depends on how it is handled. Check the products condition before using it and replace it when observing any damages.

DISPOSAL: This product disposal should be done according to the current local environmental legislation.

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