Espátula Carbonio

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Carbonio composite placement

USE INSTRUCTIONS

DESCRIPTION: Spatula for resins with special design and tips that facilitate the dental restorative procedure.

COMPOSITION: Aluminum, stainless steel and carbon coating.

INDICATION: Used to assist in the reconstruction of the dental crown, restoring its shape and function

INSTRUCTIONS FOR USE: The composite resin must be placed on the tooth, adapted, molded and sculpted using the spatula for composite resin.

CLEANING: The removal of organic matter must be done in ultrasonic vats using enzymatic detergent (minimum 3 enzymes) in dilution recommended by the manufacturer. Wash under running water. If you observe the presence of residues, repeat the process. If it persists, remove the residues through manual cleaning.

DISINFECTION: Subsequently, keep the spatula submerged in 0.2% peracetic acid for ten minutes. Dry the product.

STERILIZATION: Store in suitable packaging for sterilization. Carry out sterilization in an autoclave, at a temperature of 121°C to 135°C, observing the recommendations in the autoclave manufacturer's manual. The cleaning, disinfection and sterilization process must be repeated after each use.

DISPOSAL: The disposal of this product must be carried out in accordance with current local environmental legislation.

PRECAUTIONS: Product for professional use. Before use, check the integrity of the product. Sterilize before use. Do not touch the product to the inner walls of the autoclave.

EXPIRY DATE: 05 years from the date of manufacture, in undamaged packaging. The shelf life of this product depends on the care taken during handling.

STORAGE: Store the product in an exclusive place, in closed cabinets, protected from dust and insects.

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Rev 00 - 09.03.2023 | Cód 324510880004
ANVISA: 80213420065

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Medical device
(Europe).

European Authorized
Representative.

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