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Flexible Composite Placement

USE INSTRUCTIONS

Spatula for resins with special design and tips that facilitate dental restorative surgery.

COMPOSITION: Anodized aluminum and stainless steel. Synthetic bristles on the nº5 spatula brush.

INDICATIONS: Used to assist in the reconstruction of the dental crown, returning its shape and function.

INSTRUCTIONS FOR USE: The composite resin must be taken to the tooth, adapted, molded and sculpted with the help of a flexible spatula for composite resin.

CLEANING: The removal of organic matter must be done in ultrasonic vats using enzymatic detergent (minimum 3 enzymes) in the dilution recommended by the manufacturer. Wash in running water. If you notice the presence of residues, repeat the process. If it persists, remove residue by manual cleaning.

DISINFECTION: Afterwards, keep the spatula submerged in 0.2% peracetic acid for ten minutes. Dry the product.

STERILIZATION: Store in suitable packaging for sterilization. Carry out sterilization in an autoclave, at a temperature of 121°C to 135°C, observing the recommendations in the autoclave manufacturer's instruction manual. The cleaning, disinfection and sterilization process must be repeated with each use.

WARNING: Product for professional use. Before use, check the integrity of the product.

CAUTION: Sterilize before use. Do not touch the product on the inner walls of the autoclave.

PACKAGING: Pack the product in an exclusive place, in closed cabinets, protected from dust and insects.

SHELF LIFE OF THE PRODUCT: 05 years from the date of manufacture, in intact packaging. The useful life of this product depends on the care during its use.

DISPOSAL: This product must be disposed of in accordance with current local environmental legislation.

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