Espelho Clínico nº5

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Mouth Mirror Head

USE INSTRUCTIONS

DESCRIPTION: Clinical mirror nº5 is used to assist the dentist in the clinical visualization of the patient's oral cavity. The Front Surface mirror presents the reflective layer in the front plane (foreground) eliminating the distance between the object and the image.

COMPOSITION: Stainless steel and glass.

INSTRUCTIONS FOR USE: Attach the clinical mirror to the Indusbello mirror handle. Bring the mirror to the oral cavity for better clinical visualization.

CLEANING: The removal of dirt (organic and inorganic materials) must be done using an enzymatic detergent (minimum of three enzymes) with dilution according to the manufacturer's instructions. Cleaning can be done manually or mechanically in an ultrasonic vat. After washing the material, rinse it with plenty of water. Carry out the visual inspection and if any dirt is observed, repeat the cleaning process

DISINFECTION: Keep the product immersed in 0.2% peracetic acid for ten minutes.

STERILIZATION: Dry the product with a clean, dry cloth. Store material in suitable packaging for sterilization. Sterilization must be done by autoclave at a temperature of 121°C to 135°C, according to the autoclave manufacturer's recommendations. The cleaning, disinfection and sterilization process must be carried out after each use.

WARNING: Product for professional use.

CAUTION: Sterilize before use. Use personal protective equipment (PPE) to handle the product. Do not touch the product on the inner walls of the autoclave.

PACKAGING: Pack the product in an exclusive place, closed cabinets and protected from any contaminants.

PRODUCT SHELF LIFE: 05 years from the date of manufacture, in unbroken packaging. The useful life of this product depends on the care taken during its use.

DISPOSAL: This product must be disposed of in accordance with current local environmental legislation.

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Rev 02 - 17.01.2023 | Cód 326210880007
ANVISA: 80213420007

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Medical device
(Europe).

European Authorized
Representative.

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