Espelho Cristal

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Crystal Mirror

USE INSTRUCTIONS

Flat mirror, double-sided and in metallic crystal on both sides that provides perfect image reflection. It has rounded contours that allow a comfortable fit in the patient's mouth.

COMPOSITION: Metallized colorless crystal in stainless steel.

INDICATION: Indicated for taking intraoral photographs. Model IB 01: Lingual; Model IB 02: Lateral; Model IB 03: Pediatric Occlusal; Model IB 04: Adult Occlusal.

INSTRUCTIONS FOR USE: Choose the Mirror Cristal Indusbello according to the indication of the region to be photographed. Bring the product to the patient's mouth. Keep the product in the ideal position to carry out the photographic procedure. For better performance of the Cristal Mirror, it is recommended to use the Indusbello Cristal Mirror Handle.

CLEANING: The removal of dirt (organic and inorganic matter) must be done using enzymatic detergent (minimum of three enzymes) with dilution according to the manufacturer's instructions. Cleaning can be done manually or mechanically in an ultrasonic tank. After washing the material, rinse with plenty of water to remove any dirt and/or detergent residue. Carry out the visual inspection and if any remaining dirt is observed, repeat the cleaning process.

DISINFECTION: Keep the product immersed in 0.2% peracetic acid for ten minutes.

CHEMICAL STERILIZATION: Keep the product immersed in 0.2% peracetic acid for one hour. After the immersion time, remove the product and wash it with sterile distilled water. Afterwards, dry the mirror with a sterilized compress.

PACKAGING: Pack the product in an exclusive place, closed cabinets and protected from any contaminants.

WARNING: The Cristal Mirror cannot be sterilized in an autoclave, carry out chemical sterilization. After chemical sterilization, keep the Cristal Mirror inside the fabric envelope that comes with the product. Wool sponges and/or abrasives cannot be used to clean the mirror, as these materials result in scratches and/or stains on the piece.

PRECAUTIONS: Perform chemical sterilization before use.

EXPIRY DATE: 05 years from the date of manufacture, in undamaged packaging. The shelf life of this product depends on the care taken during handling.

DISPOSAL: The disposal of this product must be carried out in accordance with current local environmental legislation.

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