Estojo

- ESTOJOS (GERAL)
- ESTOJO 2807 - BROQUEIRO
- ESTOJO 2818 - PORTA-LIMAS

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ESTOJOS (GERAL):

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Case

- CASES (GENERAL)
- 2807 CASE FOR DRILLS
- 2818 CASE FOR ENDODONTIC FILES

INSTRUCTIONS BY MODEL:

CASES (GENERAL):

The Indusbello Autoclavable Cases allow accommodation for sterilization and material transport. They are light and easy to transport, have transparent lids that facilitate the visualization of the internal contents and exclusive safety locks. The cases have a variety of models.

COMPOSITION: Thermoplastic resin. Accessories such as profiles, supports and blankets are made of silicone.

INDICATIONS: Accommodating instruments for sterilization, storage and/or transport.

INSTRUCTIONS FOR USE: The case must be opened by deactivating the side locks, unlock it with the help of the thumb and index fingers on both sides simultaneously. After unlocking, open the lid and accommodate the materials you want to autoclave. In cases that come with the Silicone Blanket, the instruments must be placed on the blanket. In cases with profiles, the instruments must be fitted into the cavities. After placing the instruments, cover the case, place it in a suitable packaging material and carry out the sterilization process in an autoclave at a temperature of up to 135°C.

WARNING: Do not force the cover closed. Use the case in an autoclave with a temperature of up to 135°C.

process in an autoclave with a temperature of up to 135°C.

WARNING: Use the proper drill bit for high or low speed drills as needed. Respect the drill bit capacity for each case. Do not leave bits loose, keep them in the holes. Use the case in an autoclave with a temperature of up to 135°C.

2818 CASE FOR ENDODONTIC FILES:

The Indusbello file case allows the accommodation of endodontic files for sterilization, storage and transport. The file holder has nine compartments to accommodate endodontic files in a practical and safe way. The Indusbello file case has a transparent lid that allows viewing of the internal contents.

COMPOSITION: Thermoplastic resin. Note: Accessories such as profiles, supports and blankets are made of silicone.

INDICATION: Accommodate endodontic files for sterilization, storage and/or transport.

INSTRUCTIONS FOR USE: Open the case by sliding the cover upwards. Place one file in each compartment. Slide the lid down to close the case (see infographic). Pack the file holder in suitable material and carry out the sterilization process in an autoclave with a temperature of up to 135°C.

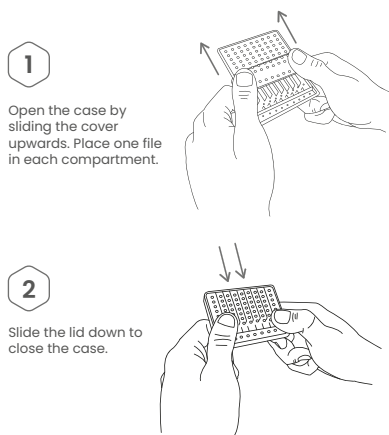
2807 CASE FOR DRILLS:

The Indusbello drill bit case allows for accommodation of drill bits for sterilization, storage and transport. The high-adaptation presenter or drill to drill model, adaptation of new drill model, adaptation of high-model or drill. The Indusbello broach case has a transparent cover that allows the visualization of the internal contents and safety locks.

COMPOSITION: Thermoplastic resin. Accessories such as profiles, supports and blankets are made of silicone.

INDICATIONS: Accommodate high and low rotation drills for sterilization, storage and/or transport.

INSTRUCTIONS FOR USE: Lift the cover to unlock it. In the holes of the support, accommodate the low or high rotation drills according to the drill bit. Close the cover until it locks. Place the closed broach case in a suitable material packaging and carry out the sterilization

WARNING: Respect the file capacity of the case. Place a file in each compartment to avoid damage to the material. Use the case in an autoclave with a temperature of up to 135°C.

GENERAL INSTRUCTIONS:

CLEANING: Clean manually with a soft sponge and neutral and/or enzymatic detergent. Wash with plenty of water.

DISINFECTION: Keep the case submerged in a 0.2% peracetic acid solution for ten minutes. Dry with a clean, dry cloth. Cleaning and disinfection must be carried out before each use.

CAUTION: Do not touch the product to the inner walls of the autoclave. Avoid contact with chemicals (solvent and chlorinated hydrocarbons) as parts may deteriorate.

PACKAGING: Pack the product in an exclusive place, in closed cabinets, protected from any dirt.

PRODUCT SHELF LIFE: 05 years from the date of manufacture, in unbroken packaging. The shelf life of this product depends on the care taken during handling.

DISPOSAL: This product must be disposed of in accordance with current local environmental legislation.

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