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Filum – Braided fiberglass impregnated with resin.

INSTRUCTIONS FOR USE:

DESCRIPTION: Braided fiberglass impregnated in composite resin light-cured.

DIMENSIONS: 85 mm X 0,2 mm.

COMPOSITION: Glass fiber yarns, diurethane, barium glass ceramics, silicon dioxide, catalysts.

INDICATION: For reinforcement of composite resin restorations, for fabrication of temporary fixed prosthesis, and for periodontal and avulsed tooth splints.

CONTRAINDICATIONS: Patients allergic to any component of the formulation.

HOW TO USE:

1. REINFORCEMENT OF COMPOSITE RESIN RESTORATION:

- 1.1 Prepare the cavity according to clinical needs;
- 1.2 Perform acid etching and apply adhesive as per the manufacturer's instructions;
- 1.3 Place a layer of composite resin and light-cure it;
- 1.4 Open the envelope containing FILUM glass fiber strips and cut a strip to fit the cavity size using scissors;
- 1.5 Place the FILUM glass fiber strip fragment in the cavity over the resin and light-cure for 40 seconds;
- 1.6 Apply another layer of composite resin, polymerize it;
- 1.7 Finish the restoration with finishing and polishing.

2. REINFORCEMENT FOR TEMPORARY FIXED PROSTHESES:

- 2.1 Create a groove 0.8 mm deep and 2 mm wide on the prosthesis;
- 2.2 Apply adhesive to the groove and light-cure as per the manufacturer's instructions;
- 2.3 Cut a piece of FILUM glass fiber with scissors shorter than the groove;
- 2.4 Fill half of the groove with composite resin, place the FILUM fiber in the groove, and light-cure for 40 seconds;
- 2.5 Apply another layer of composite resin to cover the glass fiber strip and light-cure it;
- 2.6 Finish with finishing and polishing.

3. PERIODONTAL SPLINTING:

- 3.1 Prepare the operating field with absolute isolation;
- 3.2 Create a groove 0.8 mm deep and

2 mm wide. The groove creation depends on the clinical situation;

3.3 Perform acid etching and apply adhesive as per the manufacturer's instructions;

3.4 Fill half of the groove with composite resin. Do not use microparticulate resins in splints;

3.5 Cut the FILUM slightly smaller than the groove and press it onto the composite resin;

3.6 Light-cure each segment of the fiber for 40 seconds;

3.7 Cover the groove with another layer of composite resin, extending beyond the groove limits, and light-cure it;

3.8 Remove excess resin, adjust occlusion, and finish with finishing and polishing.

HANDLING CONDITIONS: The product must only be handled by specialized professionals. Follow the product's instructions for use. The product must be disposed of in accordance with local legislation. Clean the area before using the product.

WARNINGS: Do not leave the fiber exposed. In case of exposure to the oral environment, it can be removed superficially with drills and covered with composite resin. Do not allow the structures formed by the fiber to cause occlusal interference. Correction of parafunctional habits (e.g. clenching, grinding) is recommended before using FILUM. Single use product. Reprocessing prohibited.

PRECAUTIONS: Avoid direct manual contact before polymerization. Wear gloves, mask and protective glasses. In case of accidental contact, immediately wash the area with water. If any irritation persists, seek medical assistance. Store the remaining tape in the packaging immediately after use to avoid excessive exposure to light. Restricted use to professionals specialized in dentistry, in accordance with local legislation and with specific training in its handling. Strictly follow the instructions provided in the Instructions for Use. Do not use the product after its expiration date. Keep out of the reach of children.

STORAGE: Store the product in a cool, dry place, protected from light, away from sources of excessive heat and humidity.

VALIDITY: 02 years from the date of manufacture, in undamaged packaging.

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