# Flexor de limas endodônticas

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# Endodontic files flexor

## USE INSTRUCTIONS

**INDICATION:** Manual instrument indicated for bending endodontic files.

**COMPOSITION:** Stainless steel.

**INSTRUCTIONS FOR USE:** Insert the endodontic file between the columns of the file flexor. Choose the best position to bend it according to the canal anatomy. If necessary, the file can be bent up to the tip.

**CLEANING:** Remove organic matter in ultrasonic baths, using enzymatic soap (minimum three enzymes) in the dilution recommended by the manufacturer. If residues are observed, manually wash with running water. If necessary, repeat the process.

**DISINFECTION:** Use a 0.2% peracetic acid solution for ten minutes.

**STERILIZATION:** Dry the product and package it appropriately for sterilization. Sterilize in an autoclave at a temperature of 121°C to 135°C, following the instructions in the autoclave manufacturer's manual. Repeat the process after each use.

**STORAGE:** Store the products in their original, sealed packaging, in a clean, dry place, away from direct light.

**PRECAUTIONS:** Sterilize before use. Do not allow the product to touch the internal walls of the autoclave.

**PRODUCT SHELF LIFE:** 5 years from the date of manufacture.

**DISPOSAL:** Dispose of this product according to the applicable local environmental regulations.

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