Kit CAS

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Kit CAS

USE INSTRUCTIONS

DESCRIPTION: The Indusbello CAS KIT has been specifically designed to easily and safely elevate the membrane in the maxillary sinus from the alveolar bone ridge region. The CAS KIT increases the convenience and safety of maxillary sinus surgery through safe membrane elevation. The set features a hydraulic lifting system that lifts the membrane easily and safely, as well as a bone transport system for transferring and filling bone graft materials.

COMPOSITION: Case: Polyphenylsulfone and non-toxic silicone. Instruments: Stainless steel, titanium.

RECOMMENDATION: The CAS KIT is indicated for maxillary sinus membrane lifting surgery.

CONTRAINDICATION: 1. Patients with inflammation of the sinus mucosa (sinusitis) or who have another pathology in the maxillary sinus.

2. In cases of presence of septum in the maxillary sinus in the region of interest.

3. Patients not eligible for maxillary sinus membrane elevation surgery.

METHOD OF USE: 1. From a Computed Tomography exam, determine the bone height of the alveolar ridge in the region to be installed the dental implant.

2. Incise and dilate the soft tissue, exposing the alveolar bone ridge.

3. Perform initial drilling using a Ø2.2 twist drill under copious irrigation. At this point, we recommend using the stopper to perform a perforation 1mm below the maxillary sinus membrane.

4. After the drilling sequence, the CAS bur (with the stopper ensuring that the drilling is 1 mm short of the maxillary sinus membrane) should be used to enlarge the hole, irrigating abundantly.

5. Use the depth gauge to confirm drilling.

6. Proceed with the hydraulic pressure lifting procedure using a disposable syringe (this item is not contained in the kit) and saline;

7. After lifting the maxillary sinus membrane, using the bone loader, fill the space with a particulate bone graft.

8. Use the bone condenser and spreader to compact the bone graft, repeat filling and compacting the graft as needed.

9. Proceed with the dental implant installation procedure.

CLEANING/STERILIZATION:

1st Step: Submerge the piece in enzymatic detergent in an ultrasonic vat;

2nd Step: Rinse with plenty of water to completely remove the residues from the solution.

Step 3: Dry with a clean, dry cloth.

4th Step: Perform visual inspection, noting if there were any failures in the cleaning process.

5th Step: Pack in suitable packaging and carry out the sterilization process.

STERILIZATION: The Indusbello CAS KIT is supplied non-sterile and must be sterilized before use. The Kit is reusable. The case and all other kit items are autoclavable.

DISPOSAL: For disposal, it is recommended that the device be deformed so that it is possible to clearly identify the situation of unfit for use. The damaged device must be disposed of in accordance with current local environmental legislation.

WARNINGS: 1. Only dentists may use this product. Do not use this product for any purpose other than that originally intended.

2. Periodically assess the cutting ability of piercing instruments. Do not use them if you notice loss of cutting ability.

3. During the use of perforating instruments, abundant irrigation of the bone tissue must be performed.

4. Carefully inspect the product for problems with its external appearance or if it has been dropped. If there are problems, replace it with a new one.

PRECAUTIONS

1. Read the package insert carefully before proceeding with the operation.

2. This product is a device for dental use only.

3. Do not store and/or sterilize the material without complete drying.

STORAGE: Store the product in its original closed packaging, in a clean, dry place, protected from direct light, with relative humidity of less than 95% and with a storage temperature of up to 50°C.

VALIDITY: 05 years from the date of sterilization.

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