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Scalpel Blades

Models: Carbon steel and Stainless steel

INSTRUCTIONS FOR USE

- DESCRIPTION:** Disposable scalpel blades.

COMPOSITION: Models of carbon steel and stainless steel.

INDICATION: Disposable scalpel blades are indicated for cutting tissue from the human body during surgical procedures and other applicable procedures. The device is sterilized by gamma radiation. The device is considered invasive and for transient use. The device meets intact skin and mucous membrane.

CONTRAINDICATIONS: Do not use other than surgeries. Do not use if the patient is allergic to metals. Do not use in cardiovascular, neurovascular, and ophthalmological surgeries.

HOW TO USE: Select the blade as needed. Inspect packaging for integrity and expiration date. Aseptically open the aluminum casing on the indicated side, so that the cutting side of the blade is facing upwards; With the help of a needle holder or surgical tweezers, push the blade towards the handle until they fit; Use the product according to appropriate surgical technique; After use, carefully remove the blade (using tweezers) by disconnecting it from the handle. Dispose of in an appropriate place in accordance with current legislation.

CLEANING/STERILIZATION: Product sterilized by gamma radiation. Disposable and ready-to-use. Not cleaning and disinfection methods are required.
- DISPOSAL:** Dispose in accordance with current local legislation.

WARNINGS: Read the instructions for use. The product must only be used by a qualified professional. Before use, check the integrity of the product and packaging along with the expiration date. Use the product immediately after opening the packaging. Single-use product, reprocessing is prohibited. Product sterility is not guaranteed if the packaging is damaged. Keep out of the reach of children.

PRECAUTIONS: Open the package from the opening direction to avoid injury.

ADVERSE EFFECTS: Improper use or incorrect use of the device may result in adverse events for the patient or user, such as deep cuts or injury to the user during blade assembly.

STORAGE/TRANSPORT: Store/transport away from direct sunlight and rain. Keep the product in a place with a temperature between 10°C and 40°C and humidity between 35% RH and 65% RH.

EXPIRY DATE: 05 years from the date of sterilization, in undamaged packaging.

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**Paramount Surgimed Limited**
A-106, Rilco Industrial Area, Bhiwadi - 301 019, Distt. Alwar, Rajasthan, India.

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