Mantas de Silicone

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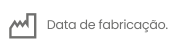
Silicone Mat for Case

USE INSTRUCTIONS

The Silicone Blanket is customized for each Indusbello case model. It has a pin-shaped surface to facilitate the accommodation of material in sterilization cases.

COMPOSITION: Non-toxic silicone.

INDICATIONS: Accommodate instruments in Indusbello sterilization cases in order to facilitate the autoclave sterilization process, preventing the materials from moving during the process.

INSTRUCTIONS FOR USE: The blanket model must be compatible with the sterilization kit. Place the silicone mat inside the case, keeping it flat. Arrange the instruments on the blanket in the best possible way, according to needs.

CLEANING: The removal of dirt (organic and inorganic matter) must be done using enzymatic detergent (minimum of three enzymes) with dilution according to the manufacturer's instructions. Cleaning can be done manually or mechanically in an ultrasonic tank. After washing the material, rinse with plenty of water to remove any dirt and/or

detergent residue. Carry out the visual inspection and if any remaining dirt is observed, repeat the cleaning process.

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PACKAGING: Pack the product in an exclusive place, closed cabinets and protected from any contaminants.

WARNING: Make sure the blanket model is compatible with the case model.

CAUTION: Carry out the cleaning and disinfection process before use. Do not touch the product to the inner walls of the autoclave.

EXPIRY DATE: 05 years from the date of manufacture, in undamaged packaging. The shelf life of this product depends on the care taken during handling.

DISPOSAL: The disposal of this product must be carried out in accordance with current local environmental legislation.

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Rev 00 - 12.01.2023 | Cód 326510880001
ANVISA Dispensado.

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Medical device (Europe).

European Authorized Representative.

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