Marcadores de Instrumentos

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MARCADORES DE INSTRUMENTOS:

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Silicone Code Rings

USE INSTRUCTIONS

SILICONE CODE RINGS: Available in round or hexagonal shapes and in many colors. Used for identification and organization of instruments during courses or clinical procedures in dental clinics where there is more than one professional. Case included.

INDICATIONS: The silicone code rings are used to differentiate instruments with a diameter from 5 mm to 9 mm, facilitating their identification and organization.

USE INSTRUCTIONS: Place the silicone code rings according to your identification needs around the instruments.

CLEANING: The organic matter removal must be performed in ultrasonic cleaner, using enzymatic soap (minimum three enzymes) in dilution recommended by the manufacturer. After washing it in running water, observe presence of residues, and repeat the process if necessary. If it persists, remove residues manually.

DISINFECTION: 10 minutes in 0.2% peracetic acid solution.

STERILIZATION: Should be done by autoclaving at a temperature of 121°C up

to 135°C (249°F up to 275°F), observing the manufacture's autoclaving instructions manual. Repeat the process for each use.

STORAGE: Keep the product in a designated and closed cabinet protected from dust and insects.

COMPOSITION: Silicone and Polypropylene.

PRECAUTION: Sterilize the product before use. Do not touch the product at autoclave's inner wall.

WARNING: Do not autoclave the silicone code rings case.

EXPIRING DATE: 05 years from manufacturing date in a non-violated package. Products durability depends on how it is handled. Mishandling it and incorrect sterilization may reduce its useful life. Check the products condition before using it and replace it when observing any damages.

DISPOSAL: This product disposal should be done according to the current local environmental legislation.

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