Moldeira

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Impression Tray

USE INSTRUCTIONS

IMPLANT TRAY: Made from colored translucent thermoplastic, in three different sizes (1,2,3). To be used in both upper and inferior dental arches. It has round edges and perforated surface that allows separating parts of the piece, according to the position of the implant to mold with open tray transferring. It has internal retention guides and vertical sections that help retaining the material that will be used during the molding process.

INDICATIONS: For either molding or protocol.

USE INSTRUCTIONS: Sterilize the impression tray before use; fill with the molding material and carry the patient's mouth. Wait for the holding time and then remove.

CLEANING: The removal of organic matter must be done in ultrasonic bowls, using enzymatic detergent (minimum of 3 enzymes) and diluted as recommended by the product's manufacturer.

DISINFECTION: Use peracetic acid at 0,2% for 10 minutes.

STERILIZATION: It must be done through autoclaving, with temperature ranging from 121°C to 135°C, observing the recommendation of the instructions of the autoclave manufacturer.

STORAGE: Dispose of the product in an exclusive place, in closed cupboards, protected from dust and insects.

COMPOSITION: Polycarbonate.

PRECAUTION: Sterilize before use. Do not touch product to the autoclave inner walls.

EXPIRY DATE: 05 years from manufacturing date, in the original packaging.

DISPOSAL: This product disposal should be done according to the current local environmental legislation.

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