Prendedor de Babador – Silicone Soft

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Bib Holder – Silicon Soft

USE INSTRUCTIONS

INDICATION: To hold hankies or bibs during dental procedures.

USE INSTRUCTION: Press the clips to open and release them at both corners of the hanky, bib or wipe to hold; press it over again to release after use.

CLEANING: The organic matter removal must be performed in ultrasonic cleaner, using enzymatic soap (minimum 3 enzymes) in dilution recommended by the manufacturer. After washing it in running water, observe presence of residues, and repeat the process if necessary. If any residue is left, remove it manually.

STERILIZATION: After visual inspection check the effectiveness of the cleaning process and pack the product in a sterilization package. The sterilization should be done by autoclaving with a temperature of 121°C to 135°C, observing the recommendations of the autoclave manufacturer's instructions manual. Repeat the process with each use.

STORAGE: Dispose the product in an exclusive place, in closed cupboards, protected from dust and insects, protected from sun and rain.

COMPOSITION: Metal and silicone.

PRECAUTION: Sterilize before use. Thoroughly dry the product before autoclaving to prevent stains. Do not touch the product to the inner walls of the autoclave."

SHELF LIFE: 05 years from manufacturing date in a non-violated package.

DISPOSAL: This product disposal should be done according to the current local environmental legislation.

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