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Rubber for Prophylaxis – Cone / Cup

USE INSTRUCTIONS

DESCRIPTION: Silicone rubbers in mandrel available in two formats (cup and cone) and two hardnesses: hard (purple) and soft (green).

COMPOSITION: Stainless steel and Silicone.

INDICATION: Recommended for prophylaxis and polishing procedures.

HOW TO USE: Select the rubber for prophylaxis according to the characteristics (shape and hardness) suitable for the case. The rubber mandrel must be fitted to the hand piece for use at low speed. After proper fitting, apply prophylactic paste to the rubber and turn on low speed. Make light and uniform movements on the surfaces of the teeth, without applying excessive force.

CLEANING/STERILIZATION: CLEANING: Dirt removal must be done using enzymatic detergent (minimum of three enzymes) diluted according to the manufacturer's instructions. Cleaning can be done manually or mechanically in an ultrasonic tank. After washing the material, rinse it with plenty of water to remove any dirt and/or detergent residue. Carry out a visual inspection and if any remaining dirt is observed, repeat the cleaning process.

DISINFECTION: Keep the product immersed in 0.2% peracetic acid for ten minutes.

STERILIZATION: Dry the product with a clean, dry cloth. Store the material in suitable packaging for sterilization. Sterilization must be done by autoclave at a temperature of 121°C to 135°C, in accordance with the autoclave manufacturer's recommendations.

DISPOSAL: Disposal of this product must be carried out in accordance with current local environmental legislation.

WARNINGS: Prophylaxis rubber must be sterilized before use. Single use is recommended. Product for professional use.

PRECAUTIONS: The professional must wear PPE and the patient must wear protective glasses while using the product. Before activating the low rotation, make sure that the prophylaxis rubber is properly fitted to the contra-angle. Do not touch the product to the internal walls of the autoclave. Before use, check the integrity of the product.

STORAGE: Store the product in an exclusive location, closed cabinets and protected from any contaminants.

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