Refil para Tamborel

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Endo Clean Stand Refill

USE INSTRUCTIONS

The refill is the foam used in the drum to pack the endodontic files.

COMPOSITION: Polyurethane.

INDICATIONS: Indicated to accommodate endodontic files in the barrel during treatment.

INSTRUCTIONS FOR USE: Place the Indusbello File Stand Refill on the bottom base of the Indusbello File Stand and fit the upper part. Insert the endodontic files.

STERILIZATION: Store the material in suitable packaging for sterilization. Sterilization must be carried out by autoclave at a temperature of 121°C to 135°C, in accordance with the autoclave manufacturer's recommendations.

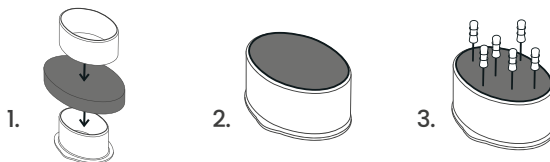
WARNING: Manufacturer recommends single use.

CAUTION: Sterilize before use. Do not touch the product to the inner walls of the autoclave.

PACKAGING: Pack the product in an exclusive place, closed cabinets and protected from any contaminants.

EXPIRY DATE: 05 years from the date of manufacture, in undamaged packaging. The shelf life of this product depends on the care taken during handling.

DISPOSAL: The disposal of this product must be carried out in accordance with current local environmental legislation.

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