Seringa Carpule Securinject

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Carpule Securinject Syringe

INSTRUCTIONS OF USE

CARPULE SECURINJECT SYRINGE: Safe dental anesthesia application product that prevents cross contamination and protects healthcare professionals from accidental needle injuries. The Syringe is composed of the applicator set (which contains the syringe body and the application rod), needle holder and needle protector. The product does not come with the needle and the anesthetic cartridge (these must be purchased separately and attached to the product by the professional before use). After the application of dental anesthesia, the professional performs the disposal of the syringe and needle through the locking system and removal of the needle protected by the protector, thus avoiding accidents and / or cross contamination by contact with fluids.

INDICATION: Carpule single use safety syringe for dental anesthesia.

INSTRUCTIONS OF USE:

- 1) Insert the anesthetic cartridge into syringe barrel.
- 2) Attach needle to syringe.
- 3) Remove needle cap and push plunger until a small amount of anaesthetic solution is expelled.
- 4) Aspirate and administer.
- 5) If syringe needs to be placed on a work surface temporarily, the needle can be covered by sliding protective sheath forward into the semi lock position.
- 6) Upon completing injection, slide protective

sheath forward, into the permanent lock position until a click is heard.

- 7) Rotate the protective sheath counterclockwise until another click is heard.
- 8) Detach protective sheath from barrel.
- 9) Dispose syringe components.

COMPOSITION: ABS polycarbonate, polypropylene, PVC pigments.

PACKAGING: Store the product in an exclusive place, in closed cabinets, protected from dust and insects. Do not expose to the sun and excessive heat.

PRECAUTIONS: Disposable product. The manufacturer recommends single use. The anesthetic cartridge and the needle must be purchased separately. Carefully follow the way the product is used to ensure that the treatment system was effective, and the needle will be properly discarded against contact and accidents. Product intended for exclusive use by healthcare professionals.

PRODUCT EXPIRY DATE: 2 years from the date of sterilization of the product. Do not use the product if it is out of date or if the packaging is damaged, damp or opened.

DISPOSAL: Disposal of this product must be carried out in accordance with the local environmental legislation in force.

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