Semplicità

Estojo Autoclavável

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Sterelization Case Autocavable

USE INSTRUCTIONS:

DESCRIPTION: *Indusbello* Autoclavable Cases allow accommodation for sterilization and transport of materials.

COMPOSITION: Thermoplastic resin.

INDICATION: Accommodate instruments for sterilization, storage and/or transport.

HOW TO USE: The case must be opened by deactivating the side locks. Open the lid, choose the internal supports with the most suitable diameter for the thickness of the instruments to be sterilized and place them in the position that provides greater firmness to the materials to be autoclaved. Accommodate the materials, cover the case, place it in packaging with suitable material and carry out the sterilization process in an autoclave.

CLEANING: Manually clean with a soft sponge and neutral and/or enzymatic detergent. Wash with abundant water.

DISINFECTION: Keep the kit submerged in a 0.2% peracetic acid solution for ten

minutes. Dry with a clean, dry cloth. Cleaning and disinfection must be carried out before each use.

DISPOSAL: The disposal of this product must be carried out in accordance with current local environmental legislation.

WARNINGS: Do not force lid closed. Use the case in an autoclave at a temperature of up to 135°C.

PRECAUTIONS: Do not touch the product to the inner walls of the autoclave. Avoid contact with chemical products (solvent and chlorinated hydrocarbons) as this may result in deterioration of the parts.

STORAGE: Store the product in an exclusive place, in closed cabinets, protected from any dirt.

EXPIRY DATE: 05 years from the date of manufacture, in undamaged packaging. The shelf life of this product depends on the care taken during handling.

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