Silano

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Silane

USE INSTRUCTIONS

DESCRIPTION: Bonding agent capable of promoting bonding between surfaces that have organic and inorganic compositions, thus increasing the bond strength between different surfaces.

COMPOSITION: Ethyl alcohol and methacryloxypropyltrimethoxysilane.

INDICATION: Recommended as a bonding agent for porcelain surfaces, ceramics and fiberglass posts.

HOW TO USE:

For cementation of indirect restorations:

Apply hydrofluoric acid according to the type of ceramic used, then wash with plenty of water and dry with an air jet. Subsequently, apply the silane to the inner surface of the ceramic using a microbrush, let it act for approximately 60 seconds and dry with light jets of oil-free air. Follow with the process of applying the adhesive and cementing the prosthetic part, according to the protocol defined in the clinical planning and the guidelines of the adhesive and resin cement manufacturers

For ceramic repairs: Prepare the substrate, according to the clinical case. Apply hydrofluoric acid to the ceramic according to the type of ceramic used, then wash with plenty of water and dry with an air jet. Subsequently, apply the silane to the ceramic surface using a microbrush, let it act for approximately 60 seconds and dry with light jets of oil-free air. Follow the adhesive application and repair protocol as defined in the clinical plan.

For fiberglass post cementation: Perform root canal preparation and conditioning and select fiberglass post. Clean the pin

with 70% alcohol and dry. Subsequently, using a microbrush, apply a layer of silane over the entire surface of the fiberglass post, wait 60 seconds and dry with an oil-free air jet. Proceed with the post cementation protocol according to the resin cement manufacturer's guidelines.

DISPOSAL: The disposal of this product must be carried out in accordance with current local environmental legislation.

WARNINGS: Product for professional use. Follow package instructions. The silane is volatile, so the bottle must be closed immediately after use to avoid evaporation of the product. Use PPE for silane handling. Avoid contact with skin and mucous membranes. In case of contact with the skin and/or mucous membranes, wash with plenty of water. Silane is flammable, therefore, it cannot be exposed to fire and/or excessive heat. The silane must not be applied on the tooth structure and/or on the external face of porcelain. Telephone for environmental or transport emergencies: 0800 110 8270 Pró-Química.

PRECAUTIONS: Keep out of reach of children. Do not use the product after the expiry date. Do not use if the package is broken.

STORAGE AND TRANSPORT: Store the product at room temperature (2° to 28°), exclusive location, closed cabinets and protected from light and any contaminants.

EXPIRATION DATE: 24 months from date of manufacture, in undamaged packaging.

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