Escova de Robinson

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Robinson Brush

USE INSTRUCTIONS

DESCRIPTION: Nylon brushes in mandrel available in three shapes (cup, flat and conical) and two hardnesses: hard (purple) and soft (green).

COMPOSITION: Stainless steel and Nylon.

INDICATION: Indicated for prophylaxis procedures.

HOW TO USE: Select the brush with the characteristics (shape and hardness) appropriate for the case. The brush chuck must be fitted to the hand piece for low speed use. After proper fit, apply prophylactic paste to the brush and turn on low rotation. Make light and uniform movements on the surfaces of the teeth, without applying excessive force.

CLEANING/STERILIZATION: CLEANING: Dirt removal must be done using enzymatic detergent (minimum of three enzymes) diluted according to the manufacturer's instructions. Cleaning can be done manually or mechanically in an ultrasonic tank. After washing the material, rinse it with plenty of water to remove any dirt and/or detergent residue. Carry out a visual inspection and if any remaining dirt is observed, repeat the cleaning process.

DISINFECTION: Disposal of this product must be carried out in accordance with current local environmental legislation.

STERILIZATION: Dry the product with a clean, dry cloth. Store the material in suitable packaging for sterilization. Sterilization must be done by autoclave at a temperature of 121°C to 135°C, in accordance with the autoclave manufacturer's recommendations. The cleaning, disinfection and sterilization process must be carried out after each use.

DISPOSAL: Disposal of this product must be carried out in accordance with current local environmental legislation.

WARNINGS: The Robinson brush must be sterilized before use. Single use is recommended. Product for professional use.

PRECAUTIONS: The professional must wear PPE and the patient must wear protective glasses while using the product. Before activating the low rotation, make sure that the brush is properly fitted to the hand piece. Do not touch the product to the internal walls of the autoclave. Before use, check the integrity of the product.

STORAGE: Store the product in an exclusive location, closed cabinets and protected from any contaminants.

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